



Manual

## Cat trap

203000

## 1 General information

It is important to always cover the trap well, for example with brush-wood, planks or similar. The floor should be covered with leaves or similar.

The trap has to be checked regularly.

Please consider legal regulations of your country.

## 2 Instructions for installation and use

Please pull the trapdoor and the bow on the left-hand side upwards. Press the leverage rod below the trap door. Now the leverage rod and the rod of the footboard have to be shifted about each other. The leverage rod has to be placed backwards, the rod of the footboard has to be placed forwards.

As soon as the cat is on the footboard, the trap door closes.

### 3 Content

Cat trap	1
1 General information	2
2 Instructions for installation and use	2
3 Content	3
4 Contact	4

GERMANY

EICKEMEYER KG  
Eltastraße 8  
78532 Tuttlingen  
T +49 7461 96 580 0  
F +49 7461 96 580 90  
E [info@eickemeyer.de](mailto:info@eickemeyer.de)  
[www.eickemeyer.de](http://www.eickemeyer.de)

ITALY

EICKEMEYER S.R.L.  
Via G. Verdi, 8  
65015 Montesilvano (PE)  
T +39 085 935 4078  
F +39 085 935 9471  
E [info@eickemeyer.it](mailto:info@eickemeyer.it)  
[www.eickemeyer.it](http://www.eickemeyer.it)

CANADA

EICKEMEYER Inc.  
250 Briarhill Dr.  
Stratford, Ont. Canada  
N5A 7S2  
T +1 519 273 5558  
F +1 519 271 7114  
E [info@eickemeyervet.ca](mailto:info@eickemeyervet.ca)  
[www.eickemeyervet.ca](http://www.eickemeyervet.ca)

NETHERLANDS

EICKEMEYER B.V.  
Bedrijventerrein  
Pavijen-West, Bellweg 44  
4104 BJ Culemborg  
T +31 345 58 9400  
F +31 345 58 9401  
E [info@eickemeyer.nl](mailto:info@eickemeyer.nl)  
[www.eickemeyer.nl](http://www.eickemeyer.nl)

SWITZERLAND

EICKEMEYER AG  
Sandgrube 29  
9050 Appenzell  
T +41 71 788 23 13  
F +41 71 788 23 14  
E [info@eickemeyer.ch](mailto:info@eickemeyer.ch)  
[www.eickemeyer.ch](http://www.eickemeyer.ch)

DENMARK

EICKEMEYER ApS  
Lysbjergvej 6, Hammelev  
6500 Vojens  
T +45 7020 5019  
F +45 7353 5019  
E [info@eickemeyer.dk](mailto:info@eickemeyer.dk)  
[www.eickemeyer.dk](http://www.eickemeyer.dk)

POLAND

EICKEMEYER Sp. z o.o.  
Al. Jana Pawła II 27  
00-867 Warszawa  
T +48 22 185 55 76  
F +48 22 185 59 40  
E [info@eickemeyer.pl](mailto:info@eickemeyer.pl)  
[www.eickemeyer.pl](http://www.eickemeyer.pl)

UNITED KINGDOM

EICKEMEYER Ltd.  
3 Windmill Business Village  
Brooklands Close  
Sunbury-on-Thames  
Surrey, TW16 7DY  
T +44 20 8891 2007  
F +44 20 8891 2686  
E [info@eickemeyer.co.uk](mailto:info@eickemeyer.co.uk)  
[www.eickemeyer.co.uk](http://www.eickemeyer.co.uk)