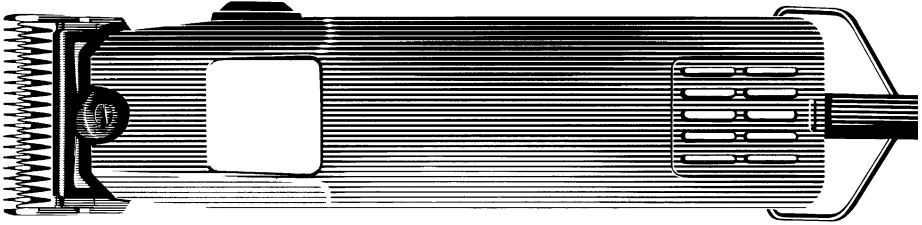
Aesculap Suhl GmbH

Favorita II, Favorita Speed, Electra® II, Ehmann turbo-line®

GB **Instructions for use/Technical description**
Shearing machine Favorita II, Favorita Speed, Electra® II, Ehmann turbo-line®

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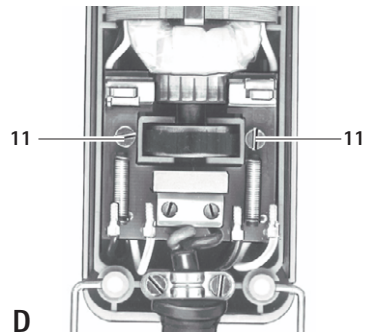
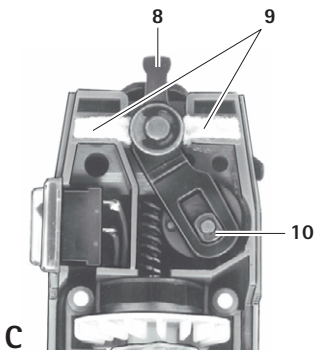
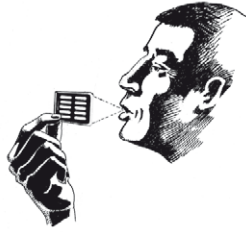
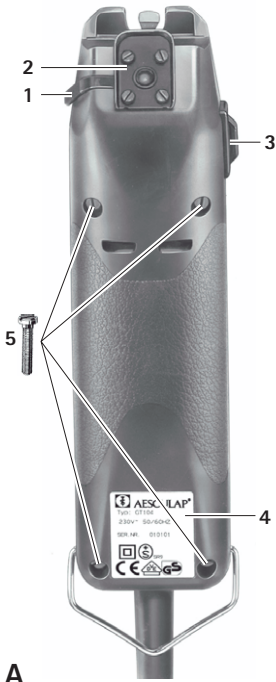
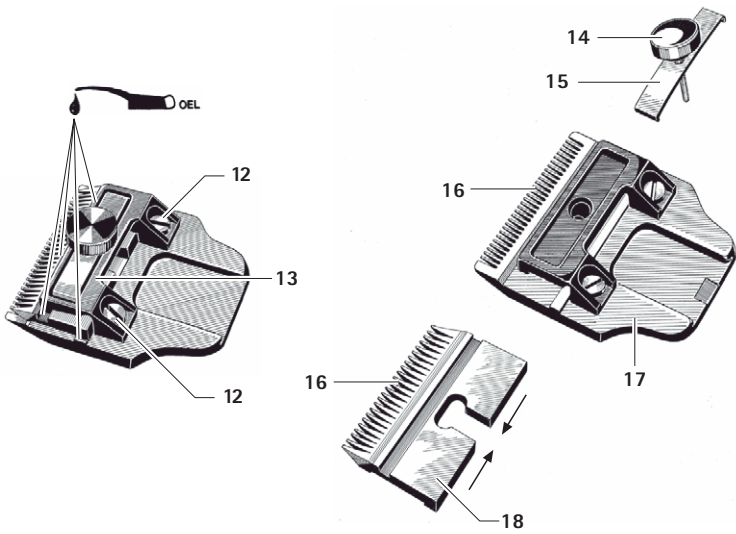
Shearing machine Favorita II, Favorita Speed, Electra® II, Ehmann turbo-line®

Legend

- 1 Lever
- 2 Shearing head adapter
- 3 On/Off switch
- 4 Nameplate
- 5 Screws (x2)
- 6 Carbon brush
- 7 Air filter
- 8 Rocker arm
- 9 Oil felts
- 10 Guide piece
- 11 Screws (x2)
- 12 Cylinder screw (x2)
- 13 Pressure piece
- 14 Knurled screw
- 15 Leaf spring
- 16 Serration
- 17 Upper cutting plate
- 18 Lower cutting plate

Symbols on product

	<p>Adhere to instructions for use</p>
	<p>Labeling of electrical and electronic devices according to directive 2012/19/EU (WEEE), see section Disposal</p>

Contents

- 1. Safe handling 3
- 2. Product description 3
 - 2.1 Scope of Supply 3
 - 2.2 Components required for operation 4
 - 2.3 Intended use 4
 - 2.4 Operating principle 4
- 3. Preparation and setup 4
- 4. Working with the device 5
 - 4.1 System set-up 5
 - 4.1.1 Connecting the accessories..... 5
 - 4.1.2 Attaching the Blade..... 5
 - 4.2 Function Check 5
 - 4.3 Safe handling Shearing head 5
 - 4.3.1 Adjusting the plate pressure..... 5
 - 4.3.2 Sharpening the cutting plate 5
 - 4.3.3 Disassemble the shearing head..... 5
 - 4.3.4 Lubricating the shearing head 6
- 5. Reprocessing 6
 - 5.1 Cleaning/disinfection 6
 - 5.2 Control, care and inspection 7
- 6. Maintenance 7
- 7. Troubleshooting list 8
- 8. Technical Service 9
- 9. Accessories/Spare parts 9
 - 9.1 Shearing heads 9
- 10. Technical Data 10
 - 10.1 Shearing machine Favorita II 10
 - 10.2 Shearing machine Favorita Speed 10
 - 10.3 Shearing machine Electra® II 10
 - 10.4 Shearing machine Ehmann turbo-line® 10
 - 10.5 Declaration of conformity 10
- 11. Disposal 10
 - 11.1 For EU countries only 10
- 12. 2-Year Warranty 11

1. Safe handling

Risk death by electric shock!

- Do not open the product.
- Connect the product only to a grounded power supply.

- Prior to use, check that the product is in good working order.
- Observe our „Notes on Electromagnetic Compatibility (EMC),“.
- To prevent damage caused by improper setup or operation, and in order not to compromise warranty and manufacturer liability:
 - Use the product only according to these instructions for use.
 - Follow the safety and maintenance instructions.
 - Ensure the manufacturer's approval before using accessories that are not mentioned in the instructions for use.
- Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge, or experience.
- Keep the instructions for use accessible for the user.
- Always adhere to applicable standards.
- Ensure that the electrical facilities in the room in which the device is being used conform to IEC norms.
- Unplug the device by pulling the plug, but never by pulling the power cord.
- Do not operate the product in explosion-hazard areas.
- Do not use the product if it is damaged or defective. Set aside the product if it is damaged.

Note

The device can be used by children over the age of 8 as well as by persons with reduced physical, sensory or mental capacities or with lack of experience and knowledge, if these persons are supervised or have been instructed on how to use the device safely, and if they understand the risks associated with the use of the device.

- Do not allow children to play with the device.
- Do not allow children to carry out cleaning and maintenance of the device without supervision.

2. Product description

2.1 Scope of Supply

Designation	Art. no.
Shearing machine Favorita II - 110 V to 125 V - 230 V	GT101 GT104
Shearing machine Favorita Speed - 230 V	GT114
Shearing machine Electra® II - 110 V to 125 V - 230 V	GH201 GH204
Shearing machine Ehmann turbo-line® - 230 V	GH214
Instructions for use	TA009813

Note

The device ships without any blade. Instead, there is a range of blades available for various applications!

Shearing machine Favorita II, Favorita Speed, Electra® II, Ehmann turbo-line®

2.2 Components required for operation

- Shearing machine Favorita II, Favorita Speed, Electra® II, Ehmann turbo-line®
- Shearing head

2.3 Intended use*Note*

The device is intended exclusively for cutting animal hair.

The shearing machine Favorita II, Favorita Speed, Electra® II, Ehmann turbo-line® is used for shearing small animals.

The device is used with a shearing head to be selected according to the respective application.

2.4 Operating principle

The device is switched on (off) by pushing the ON/OFF switch up (down).

3. Preparation and setup

Non-compliance with these rules will result in complete exclusion of liability on the part of Aesculap.

- Always observe the following guidelines when setting up and operating the product:
 - national regulations for installation and operation,
 - national regulations on fire and explosion protection.
 - operating advisories according to IEC-/VDE regulations.

**DANGER****Fire and explosion hazard!**

- **Do not operate the product in explosion-hazard areas.**

**CAUTION****Malfunction of, or damage to the product caused by improper setup!**

- **Do not expose the product to direct sunlight or moisture.**
- **Do not cover the ventilation slots of the product.**
- **Connect the product to the specified voltage of 100 V to 240 V/50 Hz to 60 Hz.**
- **Ensure that the mains power socket used for supplying the product is freely accessible.**

4. Working with the device

4.1 System set-up

4.1.1 Connecting the accessories

Combinations of accessories that are not mentioned in the present instructions for use may only be employed if they are specifically intended for the respective application, and if they do not compromise the performance and safety characteristics of the products.

- Please contact your B. Braun/Aesculap Partner or Aesculap Technical Service – Address: see section Technical Service.

4.1.2 Attaching the Blade

Risk of injury caused by sharp cutting plates!

- **Handle the shearing head of the device with appropriate caution.**

Note

The device ships without any blade. Instead, there is a range of blades available for various applications!

- Slide On/Off switch **3** upward to switch on the device.
- Push lever **1** towards the shell of the product.
- Mount the shearing head on shearing adapter **2**.
- Release lever **1**.

This completes the mounting of the shearing head.

4.2 Function Check

- Carry out a visual inspection.
- Check the switch position.
- Mount the blade correctly and lubricate it if necessary, see section Safe handling Shearing head.
- Keep a charged Li-ion battery on hand, as reserve.
- Clean the battery-powered shearing machine Favorita_{CL} before use, see section Cleaning/disinfection.

4.3 Safe handling Shearing head

4.3.1 Adjusting the plate pressure

Note

The plate pressure can be adjusted to increase the cutting power as required (see foldout page)!

- Tighten knurled screw **14** so that upper cutting plate **17** can be moved by hand, by applying moderate, lateral pressure.

4.3.2 Sharpening the cutting plate

Note

For sharpening the cutting plate, please contact Technical Service, see section Technical Service.

4.3.3 Disassemble the shearing head

Risk of injury caused by sharp cutting plates!

- **Handle the shearing head of the device with appropriate caution.**

- Loosen knurled screw **14** with leaf spring **15**.
- Push upper cutting plate **17** under pressure piece **13** out of the shearing head.

This completes the mounting of the shearing head.

Shearing machine Favorita II, Favorita Speed, Electra® II, Ehmann turbo-line®

4.3.4 Lubricating the shearing head

- Disassemble the shearing head, see section Disassemble the shearing head.
- Clean upper cutting plate **17** and lower cutting plate **18** with a soft wipe or brush. Make certain the gaps in serration **16** are cleaned properly.
- Slightly lubricate upper cutting plate **17** and lower cutting plate **18** with special lubricant for shearing heads GT604.

5. Reprocessing**CAUTION**

Damage to, or destruction of the product caused by mechanical cleaning/disinfecting or sterilization!

- Only clean and disinfect the product manually.
- Do not sterilize the product under any circumstances.

5.1 Cleaning/disinfection

Note

Always follow disposal and hygiene instructions and regulations!

Cleaning/disinfection**DANGER**

Risk of electric shock and fire hazard!

- Unplug the device before cleaning.
- Ensure that no fluids will penetrate the product.
- Leave the product exposed to air for at least 1 minute after cleaning/disinfecting.

**CAUTION**

Damage to the product due to incorrect cleaning or inappropriate cleaning/disinfecting agents!

- Use any commercial detergent for surface cleaning.
- Never immerse the product in water or detergent.
- Do not introduce any fluid into the charging bays.
- Clean the contacts in the charging bays, applying extreme care.

- Wipe the shell of the product with a lint-free cloth moistened with a commercial disinfectant for wipe disinfection.
- If necessary, after the appropriate exposure time, wipe off any residues of the cleaning/disinfecting agent with a lint-free cloth moistened with clean water.
- Dry the product with a clean, lint-free cloth.
- Clean the contacts in the charging bay with isopropanol or ethyl alcohol and a cotton swab. Do not use corrosive chemicals.
- If necessary, repeat the cleaning/disinfecting process.

Cleaning the air filter

Note

The air filter must be cleaned once a week or whenever necessary!

- Slide back and remove air filter **7**.
- Blow through or wash air filter **7**.
- Slide dry air filter **7** into its seating.

5.2 Control, care and inspection

- Allow the product to cool down to room temperature.
- Inspect the product after each cleaning and disinfecting cycle to be sure it is: clean, functional, and undamaged.
- Lubricate the shearing head as appropriate, see section Lubricating the shearing head.
- To enhance the cutting power, adjust the plate pressure as required, see section Adjusting the plate pressure.
- Check the product for any damage, abnormal running noise, overheating or excessive vibration.
- Inspect the shearing head for broken, damaged or blunt cutting edges.
- Set aside the product if it is damaged.

6. Maintenance

To ensure reliable operation, Aesculap recommends servicing the product at least once a year.

For the respective services, please contact your national B. Braun/Aesculap agency, see section Technical Service.

Service addresses

Aesculap Suhl GmbH

Fröhliche-Mann-Straße 15

98528 Suhl / Germany

Phone: +49 (0) 3681 49820

Fax: +49 (0) 3681 498234

E-Mail: suhl@aesculap.de

www.aesculap-schermaschinen.de

Other service addresses can be obtained from the address indicated above.

Shearing machine Favorita II, Favorita Speed, Electra® II, Ehmann turbo-line®

7. Troubleshooting list

Issue	Detection	Cause	Remedy
Shearing head not moving	Shearing head jammed	Shearing head not properly engaged in its seating	Properly engage the shearing head, see section Attaching the Blade
		Rocker arm defective	Replace rocker arm
Device cutting poorly or not at all	Device not cutting	Cutting edges of shearing head blunted	Have product repaired by the manufacturer
		Animal hair too wet	Only cut dry animal hair
		Shearing head lacking shearing pressure	Have product repaired by the manufacturer
	Cutting performance insufficient	Cutting edges of shearing head not honed correctly	Have product repaired by the manufacturer
	Strong heat-up of shearing head	Lack of lubrication of shearing head edges	Lubricate cutting edges of shearing head, see section Lubricating the shearing head
	Shearing head jammed	Animal hair jammed in shearing head	Demount shearing head and remove animal hair
Device motor not running	Device not cutting	Debris on electric contacts	Switch off device and clean the contacts
		Motor/Electronics defective	Have product repaired by the manufacturer
	Smell of burning from the machine housing	Motor burnt	Have product repaired by the manufacturer
	Shearing head jammed	Debris in shearing head	Clean and lubricate the shearing head, see section Lubricating the shearing head
Device motor running very slowly	Atypical motor noises	Lack of lubrication of shearing head edges	Lubricate cutting edges of shearing head, see section Lubricating the shearing head

8. Technical Service

For service, maintenance or repairs, please contact your national B. Braun/Aesculap agency.

Modifications carried out on medical technical equipment may result in loss of guarantee/warranty rights and forfeiture of applicable licenses.

Service addresses

Aesculap Suhl GmbH

Fröhliche-Mann-Straße 15

98528 Suhl / Germany

Phone: +49 (0) 3681 49820

Fax: +49 (0) 3681 498234

E-Mail: suhl@aesculap.de

www.aesculap-schermaschinen.de

Other service addresses can be obtained from the address indicated above.

9. Accessories/Spare parts

Art. no.	Designation
GT604	Lubricant
GT605	Grease
GT104614	Carbon brushes
GT104803	Air filter
GT105808	Rocker arm
GT104244	Guide piece
GH214813	Air filter (Ehmann)
TA004662	Knurled screw
TA004663	Cylinder screw for fastening the pressure piece onto the lower plate
GT730203	Leaf spring
GT730201	Plastic pressure piece

9.1 Shearing heads

Shearing head (complete)	Size EU
GH700	1/20 mm
GH703	1/10 mm
GT730	1/2 mm
GH730	turbo-line®
GH712	1,0 mm
GT736	1,0 mm
GT710	1,6 mm
GT715	2,0 mm
GT742	2,0 mm
GT746	
GT748	3,0 mm
GT754	3,0 mm
GT758	5,0 mm
GT770	7,0 mm
GT772	
GT779	9 mm
GT782	12 mm
GT784	16 mm

Shearing machine Favorita II, Favorita Speed, Electra® II, Ehmann turbo-line®

10. Technical Data**10.1 Shearing machine Favorita II**

Model	GT101 (110 V to 125 V) GT104 (230V)
Stroke rate	2 300 1/min
Protection class (acc. to IEC 60601-1)	II
Weight (without shearing head)	approx. 650 g

10.2 Shearing machine Favorita Speed

Model	GT114 (230V)
Stroke rate	3 000 1/min
Protection class (acc. to IEC 60601-1)	II
Weight (without shearing head)	approx. 650 g

10.3 Shearing machine Electra® II

Model	GH201 (110 V to 125 V) GH204 (230V)
Stroke rate	2 300 1/min
Protection class (acc. to IEC 60601-1)	II
Weight (without shearing head)	approx. 650 g

10.4 Shearing machine Ehmann turbo-line®

Model	GH214 (230 V)
Stroke rate	2 300 1/min
Protection class (acc. to IEC 60601-1)	II
Weight (without shearing head)	approx. 650 g

10.5 Declaration of conformity

	<p>We hereby declare, under sole responsibility, that this product is in conformity with the following norms or norm-defining documents:</p> <p>2004/108/EC EMC Directive DIN EN 60335-1 DIN EN 60335-2-8 Safety of household and similar electrical appliances - General Requirements - for hair cutting machines Aesculap Suhl GmbH</p>
---	---

11. Disposal

Electric tools, accessories and packaging should be handed over for environmentally responsible recycling.

11.1 For EU countries only

	<p>Do not throw electric tools in the domestic waste!</p> <p>According to European Directive 2012/19/EU on old electrical and electronic appliances and its implementation in national law, electrical tools that are no longer suitable for use must be separately collected and sent for environmentally responsible recycling.</p>
--	---

12. 2-Year Warranty

Dear Customer,

Many thanks for choosing one of our products.

For decades, the name Aesculap has stood for proven quality and first-class service. We are known to provide innovative, high-performance products for our customers. Aesculap manufactures high-quality devices and can assure you that it uses high-quality materials and careful manufacturing processes. We would like to point out to you that we are not liable for defects in our products, if these are caused by improper use, normal wear or are caused by the product being improperly used or if inappropriate or not properly functioning third-party products are used.

Parts that are typically subject to wear, and for which we are therefore not liable, are the battery and shearing head. Material defects that do not affect or only negligibly affect the value or function of the product are also excluded from our warranty.

If we are liable under our warranty, we reserve the right to repair or replace the product.

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