VISTA HF - Instrucciones

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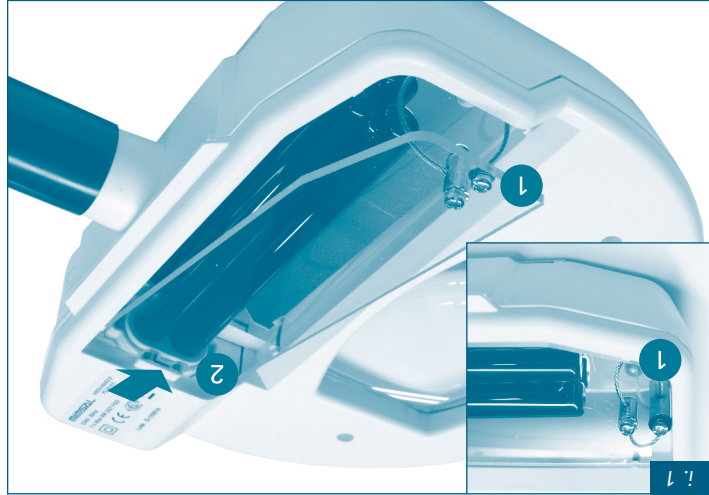
VISTA HANDLE HF - Wall bracket

VISTA HF - Instructions **ENGLISH**

VISTA HANDLE HF (handheld version) *cod. E051LAT212*
 VISTA PLUS HF (double arm articulation version) *cod. 051CAO012*

Woods lamp with magnifier.

Ultraviolet radiation lamp used specifically for dermatology, beauty treatments, veterinary care and research.

1. Installation

- Requires a MIMSAL mount, which is installed before the lamp is placed in position.
- A table clamp mount is supplied and wall or ceiling mounts or a rotating base can be supplied as optional accessories.
- MIMSAL cannot guarantee the life and functionality of the product where third party accessories are used.

2. Precautions

- Always disconnect the lamp from the power supply when cleaning or carrying out maintenance work.
- Clean either dry or using a weak solution of normal soap and water. Avoid using alcohol, acids or any type of organic solvent.
- Do not leave on for longer than 8 hours at a time.

3. Troubleshooting and maintenance

- **Maintenance:** Only for qualified electrical engineers or other qualified personnel.

To change the bulb:

- Always disconnect the lamp from the power.
- Remove the cover from the bulb (*image 1*).
- Use both hands to remove the bulb.
- Screw in the new bulb PL-S 9W/4P.
- Replace the rim cover.

Please contact your MIMSAL rep if you have any other problem.

MIMSAL only accepts responsibility for maintenance, repair or changes carried out by MIMSAL staff using original MIMSAL parts.

The manufacturer reserves the right to change the characteristics of its products without prior notice.

	VISTA HANDLE HF	VISTA PLUS HF
Light bulb	FL UV compact 9W	FL UV compact 9W
UV Wavelength	360 nanometers	360 nanometers
Mineral lens	3 diopters	3 diopters
Electrical Protection Class	Class II	Class II
Lens dimensions	10 x 6 cm	10 x 6 cm
Length of Arm	Hand, de 11 cms	72 cm
Color	White	White
Protective Cap	Plastic	Plastic
Power	220/230V	220/230V
Frequency	50/60 Hz	50/60 Hz

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