HEINE QUALITY
MADE IN GERMANY

HEINE® Headlights

CE

HEINE Optotechnik GmbH & Co. KG
Kientalstr. 7 · 82211 Herrsching · Germany
Tel. +49(0)81 52/38-0
Fax +49(0)81 52/38-202
E-Mail: info@heine.com · www.heine.com
MED 113192 2017-01-19

V-200.00.019

HEINE® Headlights

DEUTSCH	3
ENGLISH	9
FRANÇAIS	15
ESPAÑOL	21
ITALIANO	27
SVENSKA	33
NEDERLANDS	39
DANSK	45
NORSK	51
SUOMI	57
PORTUGUÊS	63

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HEINE® Headlights

These instructions apply to the following products of the HEINE® Headlight series: ML4 LED, LoupeLight 2, MicroLight 2

Please read and follow these instructions for use of and keep them for future reference. (Background: blue, foreground: white.)

Intended Use

Supplied internally with electrical power, the headlights are designed for short-term local illumination of the body, with the exception of the eyes, to support diagnosis during a therapeutic or surgical (ML4 LED only) intervention. The instrument may be used by professional medical staff and in professional healthcare institutions only.

For U.S. only:

 Federal law restricts this device to sale by or on the order of a Physician or Practitioner.

Warnings and Safety Information

 **CAUTION!** Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

 **NOTE!** Note indicates valuable advice in terms of installation, operation, maintenance or repair. Notes are important, but not related to hazardous situations.

Setting up

Only operate the **ML4 LED** with the HEINE® mPack, the mPack UNPLUGGED or the plug transformer for the headband rheostat.

Only operate the **LoupeLight 2** and the **MicroLight 2** with the HEINE® mPack mini.

Take care that the cable is fastened tightly to the headband of the ML4 LED in the provided buckles (see Fig. 1).

Fig. 1 Safe cable routing headband ML4 LED

Ensure safe cable routing (see Figs. 2 and 3) for LoupeLight 2 and MicroLight 2. Connect the cable to the lamp housing by firmly tightening the coupling nut on the cable end to the counterpart on the lamp housing.

Exposure to high mechanical strains and kinking of electrical cables on the headlights must be prevented.

Only use the HEINE plug transformer (Cat.No. X-095.16.330) for the ML4 LED.

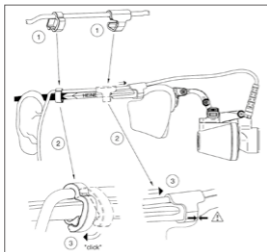
Fig. 2 Safe cable routing S-Frame

Fig. 3 Safe cable routing headband Lightweight

Adjustment of the **headband Professional L** (see Fig. 4): To adjust for the head size, the headband can be widened or tightened by turning the turnbuckles at the top (1) and the back (2).

Adjustment of the **headband Lightweight** (see Fig. 5): To adjust the head size, the headband can be widened by pressing the lever (1) and tightened by pulling at the free end (2).

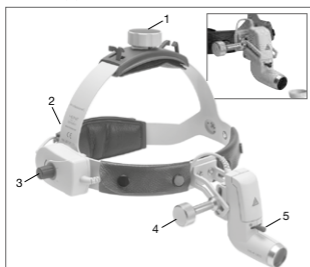
Fig. 4 Headband Professional L (see tall fig.), optical protection cover (see small fig.)

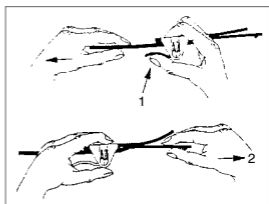
Fig. 5 Headband Lightweight

ML4 LED, fitting the headband rheostat

You can fit the headband rheostat (1) either on the left or the right side of the headband (see Fig. 6). The headband rheostat (1) can then be used for stepless adjustment of the brightness.

When using the ML4 LED, it is recommended that the mPack rheostat is set to „0“.


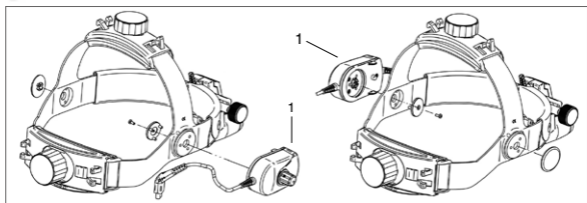
Fig. 6 Fitting the headband rheostat

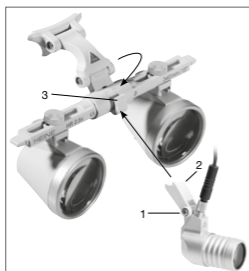How to fit the LoupeLight 2 to the binocular loupe mount with i-View® (Fig. 7):

Unscrew the screw (1) on the light mount (2) so that this can be attached to the round attachment of the loupe mount (3). The LoupeLight 2 housing has to fit between the eyepieces (not above them!).

The cable of the **LoupeLight 2** can be affixed on the right or left side piece of spectacles using the cable clips. The enclosed coloured retention clips are designed for personalisation and labelling of the loupe light.

Fig. 7 Fitting the LoupeLight 2 with i-View®

Operation

ML4 LED

To set up the ML4 LED with mPack or EN50, connect the short cord from the brightness control with the connecting cord mPack UNPLUGGED/Cinch and fix this to the back part of the headband (Fig. 8). Connect the instrument to the Cinch-socket of the mPack or EN 50. The rheostat (1) (Fig. 6) is used to adjust the brightness and to turn the device on and off.

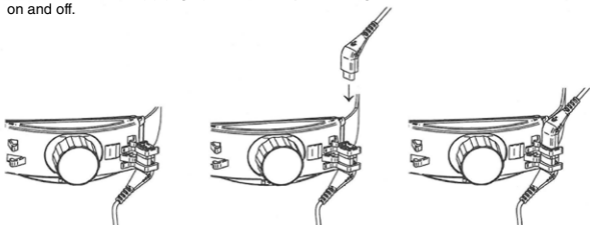
Fig. 8

To use the ML4 LED with the mPack UNPLUGGED first remove the rear cover of the headband (Fig. 9).

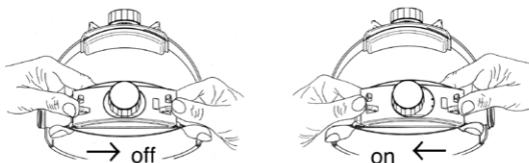
Fig. 9

Then attach the mPack UNPLUGGED to the rear of the headband. To do so, hold the headband in your left hand and push the mPack UNPLUGGED to the left, until it clicks into place (Fig.10).

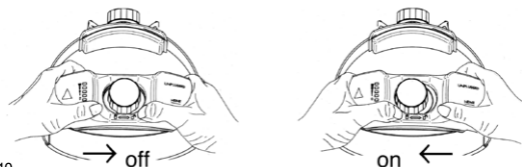
Fig. 10

Selecting viewing angle and field of view (Fig. 4)

Without S-Guard

After loosening the locking screw (4), the light head can be inclined and the angle of the light direction can be adjusted.

After adjustment, re-tighten the screw (4).

With S-Guard

Before putting on the loupes, select your inter-pupillary distance (PD) or an average value on the scale. Unscrew the retaining screw and slide the eyepieces to one side symmetrically.

Unscrew the fixing screw (4) and locate the nosepad on the bridge of the nose.

Adjust the eyepieces so that they nearly touch the protective lenses.

Shine the ML4 LED onto an object at your normal working distance and adjust the eyepieces so that the object is centred in both eyepieces. After adjustment, re-tighten the screw (4).

The loupes are correctly adjusted when you see a clearly-defined, round, shadow-free field of view. The slider (5) allows up or down adjustment of size of the light field.

LoupeLight 2 and MicroLight 2

Use the SC1 connecting cable to connect the LED light to the mPack mini (note the shape of the plug!).

Use the knob of the mPack mini to switch the LED light on and adjust the head of the light vertically such that the examination field is illuminated centrally.

Tuning the swivel lever at the mPack mini allows continuous adjustment of the brightness.

Fasten the cable on your clothes using the support clip provided (see Fig. 11).


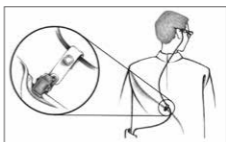
Fig. 11 Cable fastening with support clip

LoupeLight 2

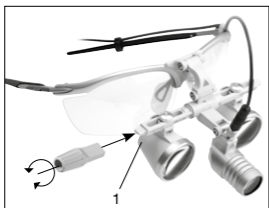Adjust the binocular loupe for the intended application. Be sure the eyepieces are positioned as symmetrically as possible on the optics holder (take off the loupe and check this), as otherwise the spot of light will appear shifted to the side.

LoupeLight 2 on loupe mount with i-View®.

If required, a steri-lever (12) can be attached to one end (1) of the loupe mount so that the binocular loupe can be swivelled independently of the light (only HR/HRP Loupes).

Fig. 12 Using the steri-lever

MicroLight 2

In order that the MicroLight 2 remains in place in every gradually and freely selectable position, it may be necessary to retighten the screws using a screwdriver (see Fig. 13).

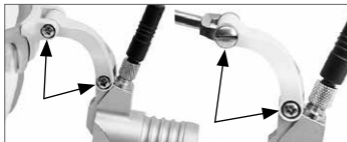
Fig. 13 Tighten the screws

Consumable material and options

For applications in the dental area:

If necessary, use a yellow filter in order to avoid the composite filling material hardening, as this blocks UV radiation.

The plug-on polarisation filter P2 enables a differentiation and low-reflex examination of tissue surfaces due to polarized light. Only in combination with ML4 LED light head or LoupeLight 2 and HR 2.5x binocular loupes, ready to use. The polarisation filter P2 is not a replacement for a yellow filter as it may harden composite filling material.

The setup and operation of the HEINE power sources are described in a separate instruction document.

Hygienic Reprocessing

Instructions on hygienic reprocessing must be adhered to, based on national standards, laws and guidelines.

Classification according to KRINKO: noncritical

Spaulding Classification USA: noncritical

 Allow the device to cool down before reprocessing.

Before cleaning, disconnect the device from the power source.

With the LoupeLight2 and the MicroLight2, leave the cable attached to the lamp. Dry the plug before you reconnect the power supply.

In the event of suspected contamination, carry out hygienic preparation of the instrument.

The described cleaning and disinfection measures do not replace the specific rules applicable for the establishment.

HEINE Optotechnik only approves the resources and procedures named in these instructions for use.

Cleaning and disinfection may only be carried out by personnel with sufficient hygienic knowledge.

Observe the instructions of the manufacturer of the reprocessing media.

Do not use any spray disinfectant, or any dripping-wet or heavily-foaming cloths. Do not prepare mechanically.

For the polarisation filter, do not use any alcoholic preparing agents.

Procedure

Clean and disinfect the head lights, cables, options and the headbands manually (wipe-cleaning and wipe-disinfecting). Clean the leather cushion of the headbands with a detergent cloth. Conditioning residues on optics may be removed using a low-bacterial, dry, micro-fibre cloth.

Recommended agents

Cleaning agent: enzymatic (e.g. neodisher® MediClean)

Disinfectant agent: quaternary ammonia compounds (e.g. Mikrobac® Tissues), agent ortho-phthalaldehyde (e.g. Cidex® OPA) or hydrogen peroxide (e.g. Accel® TB)

Number of preparation cycles

Polarisation filter may be re-prepared up to 50 times.

Please follow the cleaning notes for the S-Frame in the identically named instructions.

Maintenance and Service

The instruments do not require maintenance or service.

We as manufacturer do not stipulate a safety inspection in accordance with § 6 Medical Device Operator Ordinance (MPBetreibV), ref. attachment 1, for the HEINE product Headlights.

The expected operational life of the HEINE Headlights is 5 years.

Cable switching for LoupeLight 2 and MicroLight 2

Switch the mPack mini off by turning the controller to the zero position. Open and/or remove all cable holders such that the cable is exposed. Disconnect the plug on the mPack mini and unlock the plug on the light. Connect the new cable to the light and the mPack mini. Attach the cable to the light.

General Warnings

 Check the correct operation of the device before use! Do not use the device if there are visible signs of damage.

Do not use the device in fire- or explosive risk area (e.g. oxygen saturated or anesthetic environments)

Do not modify the device.

Use only original HEINE parts, spare parts, accessories and power sources.

The device and its accessories are not sterile. Avoid touching with sterile hands, areas or instruments.

Danger from loose, falling parts. Please check before every use that all the parts are tightened safely.

Route the supply cable in such a way that nobody can trip over it.

Do not shine the light directly into the eyes. Never ever hold magnifying glasses or any other optical system in front of the light of the LED-headlight.

Only use the lens protector as transport protection. Store the lens protector in the transport case. Do not use in the surgical rooms.

Note the operating time of the power source. You may want to have a charged spare power source ready for use.

Jerky, fast movements can alter the position of the lamp housing.

General Notes

 The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.
Store and use the device in dry and dust-free environments only!

Disposal

 The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

Electromagnetic Compatibility

Medical electric devices are subject to special precautionary measures with regard to electromagnetic compatibility (EMC). Portable and mobile high frequency communication equipment can affect medical electric devices.

 This is a device in the domestic environment, this device may cause radio interference, so that it may be necessary in this case, to take appropriate remedial measures, as e.g. orientation, new arrangement or shielding of the device or restrict the connection to the site.

The use of accessories, converters or cables other than the ones specified by HEINE might lead to increased emission reduced electrical immunity of the medical equipment.

The device may not be stacked directly near or used directly beside other devices. If the device is to be operated in a stack or with other devices, the device should be watched to ensure it operates properly in this location.

The appendix contains the tables „Guidance and Manufacturer’s declaration – Electromagnetic immunity“, technical specification and an explanation of the used symbols.

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Guidance and manufacturer's declaration – electromagnetic emissions

The device is intended for use in the electromagnetic environment specified below.
The customer or the user of the device should assure that it is used in such environment.

Emission test	Compliance	Electromagnetic environment – Guidelines
RF emissions CISPR11	Group 1	The device uses RF energy only for its internal function. Therefore, RF-emission is very low and it is unlikely that any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The device is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic Emissions IEC 61000-3-2	not applicable	
Voltage Fluctuations/ Flicker Emissions IEC 61000-3-3	Passed	

Guidance and manufacturer declaration - Electromagnetic immunity

The device is intended for use in the electromagnetic environment specified below.
The customer or the user of the device should assure that it is used in such environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – Guideline
Electrostatic discharge (ESD) acc. to IEC 61000-4-2	± 6 kV contact discharge ± 8 kV air discharge	± 6 kV contact discharge ± 8 kV air discharge	Floors should be wood, concrete or covered with ceramic tiles. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for mains cables ± 1 kV for input and output lines	± 2 kV for mains cables ± 1 kV for input and output lines	The supply voltage quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV voltage phase – phase, ± 2 kV voltage phase – earth	± 1 kV voltage phase – phase ± 2 kV voltage phase – earth	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% UT, (>95% dip in UT) for 1/2 period 40% UT, (60% dip in UT) for 5 periods 70% UT, (30% dip in UT) for 25 periods <5% UT, (>95% dip in UT) for 5 seconds	<5% UT, (>95% dip in UT) for 1/2 period 40% UT, (60% dip in UT) for 5 periods 70% UT, (30% dip in UT) for 25 periods <5% UT, (>95% dip in UT) for 5 seconds	Mains power quality should be that of a typical commercial or hospital environment. If the user of the device requires continued operation during power mains interruptions, it is recommended that the device be powered by a UPS (uninterruptible power supply) or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

Comment: UT is the a.c. supply voltage prior to application of the test level.

Guidance and manufacturer's declaration – electromagnetic immunity

The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment Guidelines
Conducted RF IEC 61000-4-6	3 Veff 150 kHz to 80 MHz	3 V eff	Portable and mobile RF communication equipment should be used no closer to any part of the device, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.
Radiated HF IEC 61000-4-3	3 V/m 80MHz to 2,5GHz	3 V/m	<p>Recommended separation distance: $d = 3,5/3 * \text{SQRT}(P/W)$ $d = 3,5/3 * \text{SQRT}(P/W)$ 80 MHz to 800 MHz $d = 7/3 * \text{SQRT}(P/W)$ 800 MHz to 2,5 GHz</p> <p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey^a, should be less than the compliance level in each frequency range.^b</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> 

Note 1: At 80Hz and 800MHz, the higher frequency range applies.

Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a Field strengths from fixed transmitters, such as base stations for radio (cellular / cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy.

To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the device is used exceeds the applicable RF compliance level above, the device should be observed to verify normal operation.

If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the device.

b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V / m.

Recommended separation distances for portable and mobile RF communication equipment and the device

The device is intended for use in the electromagnetic environment in which radiated RF disturbances are controlled. The customer or user of the device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communication equipment (transmitters) and the device as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter (W)	Separation distance according to frequency of transmitter (m)		
	150 kHz to 80 MHz $d = 3,5/3 * \text{SQRT}(P)$	80 MHz to 800 MHz $d = 3,5/3 * \text{SQRT}(P)$	800 MHz to 2,5 GHz $d = 7/3 * \text{SQRT}(P)$
0.01	0.1	0.1	0.2
0.1	0.4	0.4	0.7
1	1.2	1.2	2.3
10	3.7	3.7	7.4
100	11.7	11.7	23.3

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be determined using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

Note 1: At 80 MHz and 800 MHz, the higher frequency range applies.

Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Technical specification

	ML4 LED	LoupeLight 2 MicroLight 2
Nominal voltage	6-9 V DC	3 V DC
Nominal current	max. 600 mA	max. 350 mA
Protection class	II (power supply) Internally powered (accu)	
Illuminance (without filters)	typ. 65,000 lx	typ. 55,000 lx
Colour temperature	typ. 4,500 K	typ. 4,000 K
Colour rendering index (CRI)	typ. ≥ 90	
Cable length	approx. 1.60 m	
Light field diameter at a 420 mm working distance	30-80 mm	approx. 80 mm
Vertical direction of illumination	- 20° to $\geq 20^\circ$	- 90° to $\geq 90^\circ$ - 20° to $\geq 20^\circ$
Weight	approx. 70 g (light head rheostat)	approx. 50 g (only light) approx. 80 g (with S-Frame) approx. 190 g (with Lightweight Headband)
Operation time	typ. 8.5 h (with mPack) typ. 3.5 h (with mPack UNPLUGGED)	typ. 4 h (with mPack mini)
Environmental conditions for operation	+10 °C to +35 °C 30 % to 75 % rel. humidity 700 hPa to 1060 hPa	
Environmental conditions for storage	+5 °C to +45 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa	
Environmental conditions for transport	-20 °C to +50 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa	
Device classification according to IEC 62471	Group 2	

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Explanation of utilized symbols

The following symbols are used on the device or on the packaging:

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