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ANIMAL PAD
ELECTRICALLY HEATED

USER MANUAL

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TELEPHONE +49 7461 96 580 0

www.eickemeyer.com

veterinary technology for life
Eickemeyer®

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CONTENT

1. Important Instructions for Use	4
2. Directions for Use	4
3. Tips for Use	4
4. Connecting up to the Electricity Supply	5
5. Cleaning	5
6. Servicing and Repair	5
7. Guarantee	5

1. IMPORTANT INSTRUCTIONS FOR USE

This pad is designed to supply a gentle heat to an animal whilst upon it. As this is only a gentle heat any warmth will dissipate as soon as it is created, and you may in fact be excused from thinking it is not operating correctly, especially if you place your hand on top and can feel no immediate heat. If you require to test the pad we suggest placing a heavy object such as a large book on top for approx. 30 minutes a slight heat will then start to build.

2. DIRECTIONS FOR USE

Place the pad where it is to be used making sure it is kept flat. On initial use check heating regularly see "Tips for Use". Leave plugged all the time heat is required.

3. TIPS FOR USE

The pad is contained in a vinyl waterproof cover which can be wiped clean using a damp cloth. For heavy soiling a detergent can be used. Never submerge the pad in water. If used direct on a floor that is uninsulated such as concrete we suggest placing material such as a blanket underneath to prevent heat loss through the floor. If the pad is to be covered use only a light weight material this will allow the heat to come through.

The heating pad if used continuously should be checked frequently, and when used with sick or infirm animals. Should the animal for which the pad is to be used, be liable to chew it, we suggest some precautions be taken to protect the pad. This may be done by cutting a piece of thin plywood slightly bigger than the pad wrap some heavy textile e.g. sacking or canvas fixing on the underside with the pad uppermost. Warmth will still come through the heavy layer.

If the animal for which the pad is intended for a be a persistent chewer we suggest that you may want to deprive the animal use of the pad. If you do want to use the pad and are concerned about chewing, metal guards to place over the cable can be purchased from most D.I.Y. outlets, as can a circuit breaker.

CAUTION – THE FOLLOWING MUST BE OBSERVED:

- DO NOT launder or dry clean
- DO NOT use wet
- DO NOT use folded
- DO NOT immerse in water
- DO NOT exert tension or pressure on flexible lead
- DO NOT insert pins
- DO NOT use on soft furnishings

Avoid pressing sharp creases or folds onto pad.

Examine pad regularly especially for signs of wear and damage. If there are such signs, or any other signs of damage it should be returned for service. In any case we suggest the pad is serviced every two years – see "Servicing and Repair".

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EICKEMEYER Ltd.
EICKEMEYER B.V.
EICKEMEYER Inc.