

INFUSOVET SP SYRINGE PUMP

USER MANUAL



Item no. 402095

TELEPHONE +49 7461 96 580 0

www.eickemeyer.com

veterinary technology for life
Eickemeyer®



Release date: 2022-05

Revision: 2.0

Preface

Manual Purpose

This manual contains the instructions necessary to operate the product safely and in accordance with its function and intended use. Observance of this manual is a prerequisite for proper product performance and correct operation and ensures animal and operator safety.

This manual is based on the maximum configuration and therefore some contents may not apply to your product. If you have any question, please contact us.

This manual is an integral part of the product. It should always be kept close to the equipment so that it can be obtained conveniently when needed.

Intended Audience

This manual is geared for veterinarian who are expected to have a working knowledge of medical procedures, practices and terminology as required for monitoring of animals.

Illustrations

All illustrations in this manual serve as examples only. They may not necessarily reflect the setup or data displayed on your equipment.

Conventions

- ***Italic text*** is used in this manual to quote the referenced chapters or sections.
- [] is used to enclose screen texts.
- < > is used to enclose the keys.
- → is used to indicate operational procedures.

Contents

1 Safety	1-1
1.1 Safety Information	1-1
1.1.1 Dangers	1-1
1.1.2 WARNING	1-2
1.1.3 CAUTION	1-3
1.1.4 NOTE	1-3
1.2 Equipment Symbols	1-5
2 Overview	2-1
2.1 Description	2-1
2.1.1 Indications for Use	2-1
2.1.2 Contraindications	2-1
2.1.3 Appearance, Parts and Features	2-1
2.2 Host	2-2
2.2.1 Front View	2-2
2.2.2 Rear View	2-4
2.2.3 Side View	2-5
2.2.4 Bottom View	2-6
2.3 Screen Display	2-6
2.4 Cursor	2-7
3 Installation and Setting	3-1
3.1 Installation	3-1
3.1.1 Out of Box Audit (OOBA)	3-2
3.1.2 Operating Conditions	3-2
3.1.3 Mount the Clamp	3-3
3.1.3.1 Standard Pole Clamp	3-3
3.1.3.2 Advanced Pole Clamp (Optional)	3-3
3.1.4 Connect the AC Power Source	3-4
3.2 Conventional Settings	3-4
3.2.1 Set Language	3-4
3.2.2 Adjust Screen Brightness	3-5
3.2.3 Set Date and Time	3-5
3.2.4 Adjust Volume	3-5
3.3 Restore Factory Default	3-5
4 Basic Operation	4-1
4.1 Infusion Flow Chart	4-1
4.2 Operational Procedures	4-2
4.2.1 Turn on the Pump	4-2

4.2.2 Insert the Syringe	4-2
4.2.3 Change the Syringe	4-3
4.2.4 Select Syringe Brands.....	4-4
4.2.5 Infusion Mode	4-4
4.2.6 Purge	4-5
4.2.7 Set Infusion Parameters.....	4-5
4.2.8 Infusion	4-5
4.2.9 BOLUS.....	4-6
4.2.10 Change the Rate during Operation	4-6
4.2.11 Complete.....	4-7
4.2.12 Standby.....	4-7
4.2.13 Turn off the Pump.....	4-7
5 Setting Parameters	5-1
5.1 KVO.....	5-1
5.2 Occlusion Pressure (unit)	5-1
5.2.1 Set Occlusion Pressure.....	5-1
5.2.2 Set the Pressure Unit.....	5-1
5.2.3 Dynamic Pressure Scanning (DPS)	5-2
5.2.4 Automatic Pressure Release Function (Anti-Bolus)	5-2
5.3 Key Lock Function.....	5-2
5.4 Reminder Function	5-3
5.5 Time Near End	5-3
5.6 Common Syringe Brands	5-3
5.7 Bed No. Settings	5-4
5.8 View Department.....	5-4
6 Other Functions	6-1
6.1 Nurse Call.....	6-1
6.2 Wireless Networking (Optional).....	6-2
6.3 Data Export	6-3
6.4 WLAN Setting.....	6-3
7 Alarms.....	7-1
7.1 Alarm Level.....	7-1
7.2 Alarm Types.....	7-1
7.2.1 Multi-level Alarm Rules.....	7-2
7.3 Alarm Handling Rules	7-2
7.4 Alarm Countermeasures.....	7-3
8 Battery	8-1
8.1 Battery Performance Optimization.....	8-2
8.2 Check the Battery	8-2
8.3 Battery Recycling	8-3

9 Preservation and Sanitation.....	9-1
9.1 Description	9-1
9.2 Cleaning	9-1
9.3 Disinfection.....	9-2
10 Maintenance	10-1
10.1 Inspection	10-1
10.2 Maintenance Plan.....	10-1
10.3 View Information.....	10-2
10.4 Syringe Calibration	10-2
10.5 Safe Disposal and Recycling	10-3
11 Accessories	11-1
A Product Specifications.....	A-1
A.1 Safety Specifications.....	A-1
A.1.1 Product Classification	A-1
A.1.2 Operating Environment	A-2
A.2 Physical Specifications.....	A-2
A.3 Hardware Specifications	A-3
A.3.1 Display	A-3
A.3.2 Battery	A-3
A.3.3 Host LED	A-3
A.3.4 Auditory Indicator	A-3
A.3.5 External Ports	A-4
A.3.6 Signal Output Interface	A-4
A.4 Wi-Fi Specifications	A-4
A.5 Specifications.....	A-5
A.6 A Reference Table Showing Occlusion Alarm Delay and Possible Dose.....	A-7
A.7 Infusion Accuracy Curve	A-8
A.8 Trumpet Curve	A-9
B EMC and Radio Regulatory Compliance	B-1
B.1 EMC.....	B-1
B.2 Radio Regulatory Compliance	B-6
C Default Factory Settings	C-1
C.1 Alarms.....	C-1
C.2 Interface.....	C-1
C.3 Parameters	C-1
C.4 System Time.....	C-2
D Alarm Information.....	D-1

E Symbols and Terms	E-1
E.1 List of Units	E-1
E.2 List of Symbols.....	E-2
E.3 List of Terms.....	E-2
E.4 List of Unit Conversion	E-3
F Toxic and Hazardous Substances or Elements	F-1

1 Safety

1.1 Safety Information

The safety statements presented in this chapter refer to basic safety information that the operator must pay attention to and abide by when using the veterinary syringe pump. There are additional safety statements in other chapters or sections, which may be the same as or similar to the following, or specific to particular operations.

Dangers

- Indicates an imminent hazard that, if not avoided, could result in death, serious injury or damage to product/property.
-
-

WARNING

- Indicates a potential hazard or unsafe practice that, if not avoided, could result in death, serious injury or damage to product/property.
-
-

CAUTION

- Indicates a potential hazard or unsafe practice that, if not avoided, could result in minor personal injury, product malfunction or damage to product/property.
-

NOTE

- Provides application tips or other useful information to ensure that you get the most out of the product.
-

1.1.1 Dangers

This Manual does not contain any information at the "Danger" level.

1.1.2 WARNING



- Device, cables and accessories must be inspected before use to guarantee their normal and safe operation.
- This equipment can only be connected to the socket with ground protection. Please adopt a rechargeable battery instead of the socket as the power supply if the socket is not provided with a ground lead.
- To prevent fire or explosion, do not operate the equipment in the presence of anesthetic, flammable or explosive materials.
- Do not open the equipment casing as there is the impending danger of electric shock. Equipment maintenance and upgrades must be carried out by maintenance technicians whom are trained and licensed by the manufacturer. Moreover, the process must be done only after the AC power supply is disconnected. Maintenance carried out by individuals non-affiliated to the manufacturer or by non-licensed personnel may affect the safety, performance and function of the product.
- When used with electrosurgery equipment, the safety of animals should be ensured.
- The animal's condition and the working condition of the veterinary syringe pump must be monitored carefully, and the alarm volume and alarm levels need to be set according to the actual needs. Operation and performance relying solely on the auditory alarm system alone is not sufficient, and setting the alarm at a low volume may endanger the animal. If the alarm volume is less than the surroundings volume, which can further lead to operators identify alarm mistakenly.
- Please carefully install the power line and cables with various accessories to prevent the animal from choking or suffocation caused by entanglement of the cables or by electrical disturbance.
- The packaging materials must be disposed of in compliance with local laws and regulations or the pet hospital policy on waste management. They must be kept out of the reach of children.
- Infusion cannula knots, filter coagulation and occlusions arising from needle insertion can cause the pressure inside the syringe to rise during infusion. When this occurs, removing the occlusion can cause excessive liquid to be infused into the animal, so appropriate measures should be taken.
- This equipment has to be used with professional medical consumables, and its accuracy cannot be guaranteed when it is used with a syringe which is a non-standard consumable or a consumable without

calibration, please contact the company for calibration service.

- Do not touch the animal when connecting the peripheral equipment via the input/output signal ports to prevent animal leakage current from exceeding the requirements specified by the standard.
 - In the process of defibrillation, do not touch animal and other non-defibrillation equipments to prevent electric shock damage, and defibrillation will not affect the basic performance (such as infusion accuracy, alarm and signal transmission) of the pump.
-
-

1.1.3 CAUTION



- Use the accessories specified in this Operator's Manual to guarantee the animal's safety.
 - When equipment and the accessories exceed their recommended service life, they must be disposed of in accordance with local statutes or pet hospital regulations.
 - Electromagnetic fields may affect equipment performance. This makes it necessary for other equipment used in the vicinity of the pump to meet EMC standards. Mobile phones, X ray and MRI equipment are all potential interference sources because of their high-intensity electromagnetic radiation.
 - Before the equipment is connected to the power supply, check that the voltage and frequency of the power supply match the specifications on the label or in this Operator's Manual.
 - Please install and carry the equipment correctly to protect the equipment from damage from drops, impacts, violent shaking or other external mechanical forces.
 - Disposable accessories must be disposed of after use in accordance with the relevant pet hospital regulations.
 - Avoid direct sunshine, high temperatures and dampness.
 - Check the built-in battery before use to make sure it has sufficient power. Recharge the battery if necessary.
 - The syringe with the luer taper is recommended for use, which can effectively prevent animals from under current caused by the occurrence of the cannula to slip out when under tension.
-

1.1.4 NOTE

NOTE

- Install the equipment in a position where it can be easily accessed for inspection, operation and maintenance.
 - Keep this Operator's Manual near to the equipment for future ease of reference.
 - The software of the equipment is developed according to the software development demands of IEC60601-1 standard, which can minimize the possibility of the risk caused by program error.
 - This Operator's Manual describes the most complete functional configuration of the equipment. The product you are using may not have some of the settings or functions described herein.
 - Do not insert devices that are not specified by the manufacturer into the multifunction interfaces.
 - During infusion, the veterinary syringe pump can accurately control the rate, infusion volume and infusion time, and monitor the operation in real-time, to effectively prevent over currents, under currents and instances of backflow.
 - The device is not in touch with the drugs or animals directly. Thus, there is no need to process Biocompatibility test on it.
-

1.2 Equipment Symbols

The equipment you purchased may not provide you with all the following symbols.

	Consult instructions for use		ON/OFF
	Caution		Syringe
	Alternating current power supply (AC)		Battery
	Alarms		AUDIO PAUSED
	Clear/Back		Start
	Bolus		Confirm
	Stop		Menu
	Move up/Increase		Move down/Decrease
	Move left		Move right
	Configured wireless module and connected successfully		Wireless transceiver
	Lock		Multifunction interface
	Serial number		Protected against defibrillation CF applied parts
	Date of manufacture		Manufacturer
	This way up		Keep dry
	Fragile, handle with care		Stacking limit by number

	NRTL certification mark	IP34	Protected against solid foreign objects with a diameter no less than 2.5mm and protected against spraying liquid water
	CE mark		Recycle
	Environmentally-friendly use periods of electronic products (20 years)		Temperature limit
	Atmospheric pressure limitation		Humidity limitation
	<p>The following definition of the WEEE label applies to EU member states only: the use of this symbol indicates that waste electrical and electronic equipment must not be disposed of as unsorted municipal waste and must be collected separately. By ensuring that this product is disposed of correctly, you will help prevent bringing potential negative consequences to the environment and human health. For more detailed information with regard to returning and recycling this product, please consult the distributor from whom you purchased the product</p>		

2 Overview

2.1 Description

2.1.1 Indications for Use

The veterinary syringe pump is used in conjunction with the syringe to control the dose of liquid infused into the animal's body.

This veterinary syringe pump is expected to be used in institutes or units with healthcare capabilities.



WARNING

- **The veterinary syringe pump is used for accurate and continuous infusion to animals. It must only be used under appropriate conditions by professional veterinarians, or by suitably trained veterinarians. Personnel using this product must receive sufficient training. This product must not be operated by anyone who has not been authorized to do so or has not received suitable training.**
-

2.1.2 Contraindications

None


2.1.3 Appearance, Parts and Features

The veterinary syringe pump primarily consists of a housing, and pole clamp. By precisely controlling the rotational speed of the stepping motor, the screw rod is driven to run at the set speed, the dose of liquid infused into the animal's body by syringe can be controlled, the veterinary syringe pump can be used for precise and continuous infusion of liquids, and all components are suitable for use in animal environment. Wireless modules are optional. Functions of the software comprise Rate Mode and Anti-bolus function.

Since some parts and functions are optional, the veterinary syringe pump you purchased may not contain these additional parts and their relevant functions.

2.2 Host


2.2.1 Front View

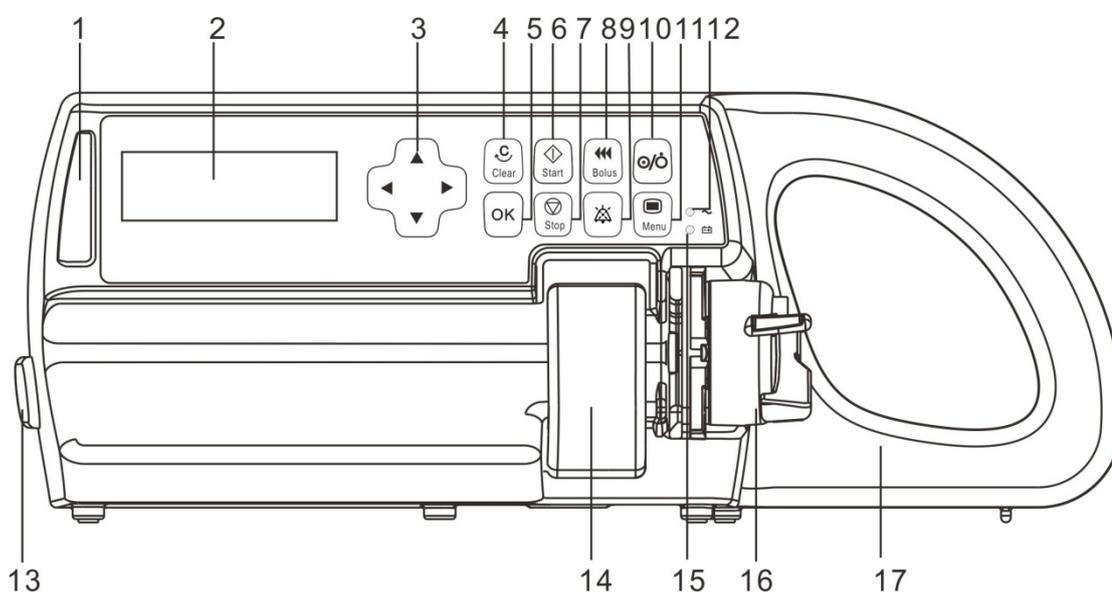

1.	Alarm light The alarm light indicates different alarm levels in different colors and flash frequencies, please refer to Chapter 7 Alarms for details.
2.	Display Used for displaying infusion parameters and relevant content.
3.	<DIRECTION> Used for adjusting value, change lines and pages.
4.	<CLEAR/BACK> <ul style="list-style-type: none"> ■ Under non-setting status, indicate to return to the previous menu or operation. ■ Under the setting status, indicate to clear the current set or cancel the edit.
5.	<OK> Used for confirming input operation and saving value.
6.	<START> After installing the syringe correctly and completing setting infusion parameters, press this key to start the infusion.
7.	<STOP> During infusion, press this key to stop infusion. Infusion stops caused by alarms, such as occlusion and so on, press this key to cancel the alarm.


8.	<p><BOLUS></p> <ul style="list-style-type: none"> ■ During infusion, press this key to start fast infusion. ■ When the pump is stopped, press this key to purge.
9.	<p><AUDIO PAUSED></p> <p>Pauses alarm sound.</p>
10.	<p><POWER></p> <ul style="list-style-type: none"> ■ Used for turning power on, entering in standby state and turning off operations. ■ When power off, press and hold (>3 s) the key.
11.	<p><MENU></p> <ul style="list-style-type: none"> ■ Under non-operation status, used for switching [Main Menu] interface and other interfaces. ■ Under operation status, press and hold this key to lock; in locked state, press and hold to unlock.
12.	<p>AC/DC indicator light</p> <ul style="list-style-type: none"> ■ On: The pump is connected to an AC/DC power supply (including shutdown). ■ Off: The pump is not connected to an AC/DC power supply.
13.	<p>Extension cannula clamp</p> <p>Fixes the extension cannula.</p>
14.	<p>Syringe fixation clamp</p>
15.	<p>Battery indicator</p> <ul style="list-style-type: none"> ■ Steady green indicates that the battery is charging (including shutdown). ■ Flashing indicates that the battery is providing power. ■ Light off indicates that there is no battery or the pump is turned off and not connected to an AC power supply.
16.	<p>Slider</p> <p>Secures syringe and drives plunger assembly.</p>
17.	<p>Handle</p>

2.2.2 Rear View

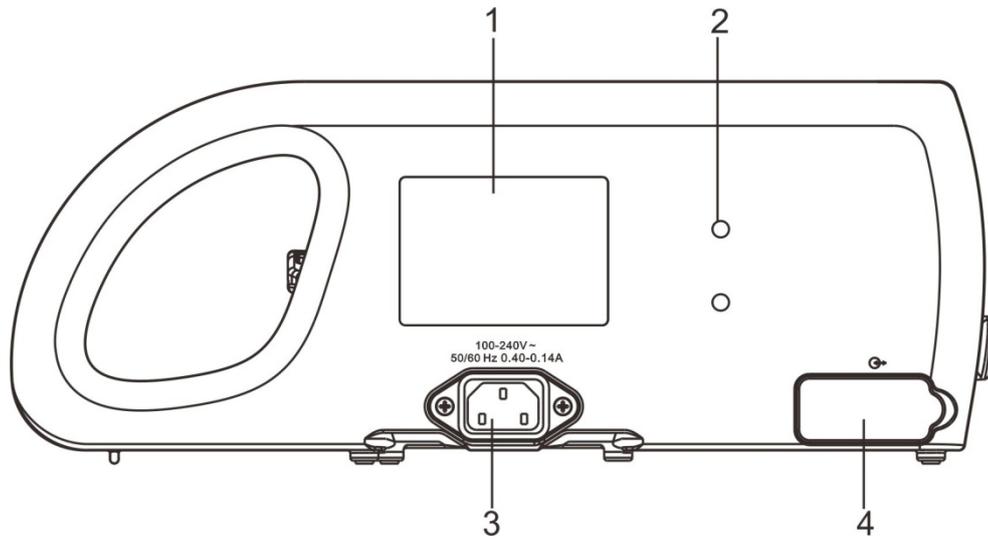


1.	Product label
2.	Pole clamp mounting holes (two)
3.	Alternating current power supply (AC) port Connected by three-core-type power cord and AC power source.
4.	Multifunction interface, which combines the following interface functions: <ul style="list-style-type: none">■ DC power input interface■ RS232 interface■ Nurse call interface

2.2.3 Side View

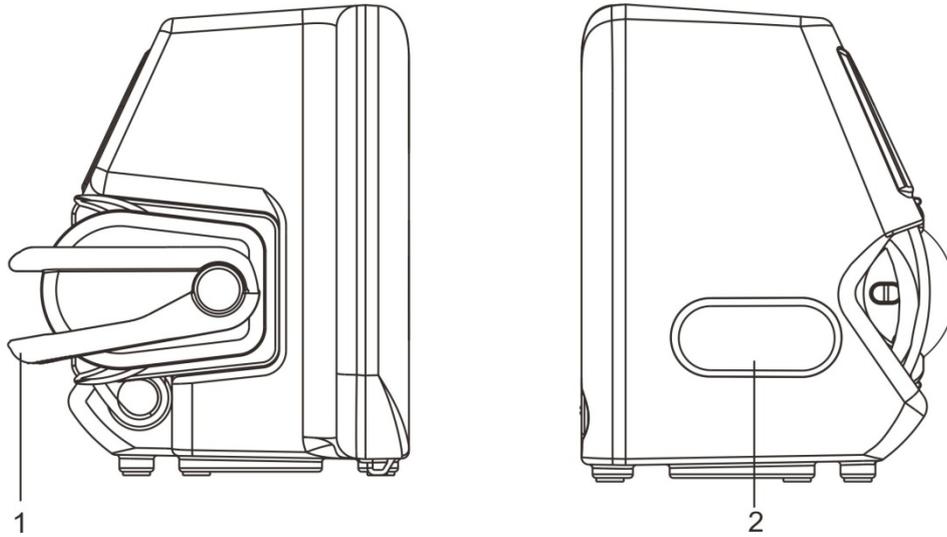


1.	Handle
2.	Extension cannula clamp

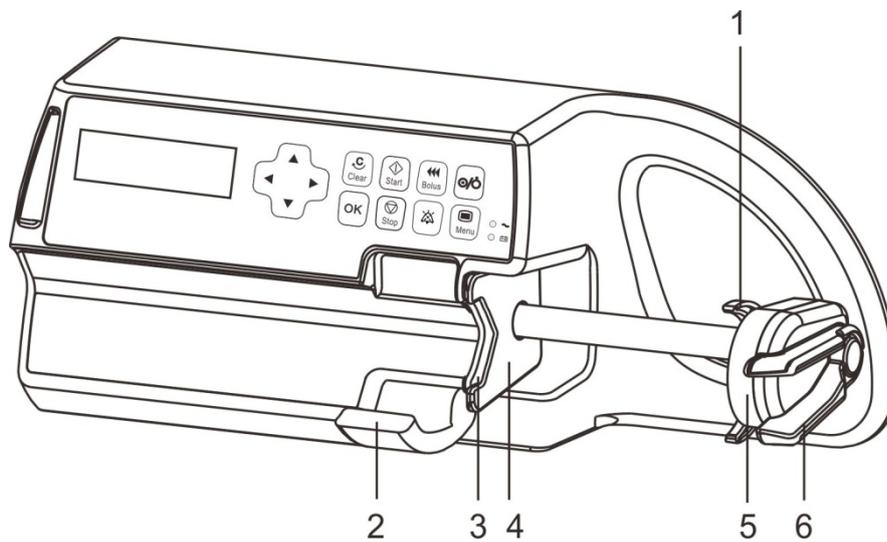

1.	Clip
2.	Syringe fixation clamp
3.	Slot
4.	Spindle clamp
5.	Slider
6.	Handle


2.2.4 Bottom View

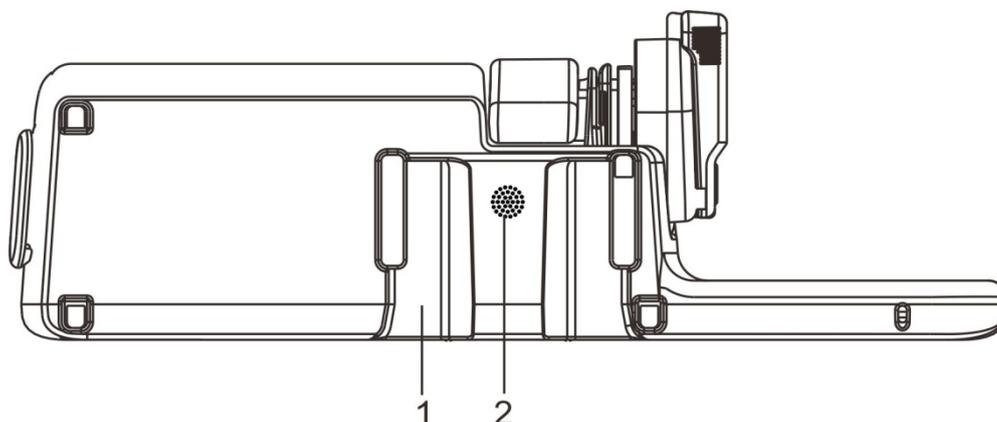

1.	Multi-channel pumps connection slot
2.	Speaker hole

2.3 Screen Display

This veterinary syringe pump has a monochrome LCD screen. The display information comprises two main parts:

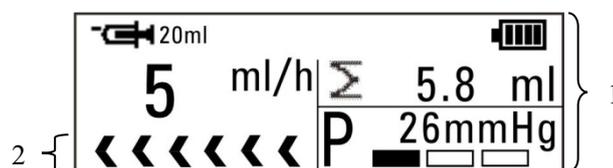

1.	<p>Parameter area</p> <p>Display syringe icon, battery icon, every parameter and the parameter value of the current screen, etc.</p>
2.	<p>Prompt bar</p> <p>Display run icon. The run icon on the screen displays the running operation:</p> <ul style="list-style-type: none"> <p>▪ </p> <p>The icon indicates normal running. Arrows move from right to left, and the running speed increases as the rate is increased.</p> <p>▪ </p> <p>Motor stops caused by alarms during infusion, no icon.</p>

2.4 Cursor

In the main screen and parameter settings screen, when the cursor is located at an option or at a data value, the grounding of the option or the data value will turn to white and the font will become blue. Press  or  to move cursor up or down and confirm the location. Press  to select the option or data value for further operation.

Note: Press  or  to "locate" cursor; Press  to "Select".

FOR YOUR NOTES

3 Installation and Setting

3.1 Installation

WARNING

- Equipment assembly and refit (including correct protective grounding connection) during life period must be carried out by maintenance technicians whom are trained and licensed by the manufacturer, and evaluated according to the specified IEC60601-1. Please contact the company if you have any queries.
 - The software copyright for this equipment belongs to the manufacturer. Unless explicitly authorized, any alteration, reproduction or sale by any means or in any form by any organization or individual is prohibited.
 - All the analog equipment and digital facilities should be certified according to the specified IEC standard (such as: IEC60950 Information Technology Equipment Safety and IEC60601-1 Medical Electrical Equipment Safety). Moreover, all equipment should be connected based on the requirements of the valid version of the IEC60601-1 system. The qualified individual responsible for connecting auxiliary equipment to the input and output signal ports is also accountable for making the system in accordance with the IEC60601-1 standard. Please contact the company if you have any queries.
 - When this equipment combining with other electrical equipments forms a combination with a special function, and the user cannot determine whether there is an impending danger from each equipment specification (such as a danger of electric shock due to aggregation of current leakage), please contact the company or a specialist in the field at the pet hospital, to guarantee that all equipment in the combination are safe enough and will not be damaged.
 - Please make sure this equipment is securely fixed and positioned. Positioning changes and severe shock can lead to minor changes in the delivery accuracy.
-
-

NOTE

- Removing power cord is to disconnect equipment from power supply. Please ensure suitable clearance around the equipment to facilitate connect and remove power cord.
 - This equipment is in accordance with the EN 1789:2007+A1:2010 standard. It can be fixed on cross bar (square cross bar: 10 x 25 mm) or vertical bar (round vertical bar with diameter size of 15-38mm) of ambulance with advanced pole clamp. Please refer to 3.1.3.2 *Advanced Pole Clamp (Optional)* for detailed operation of advanced pole clamp
-

3.1.1 Out of Box Audit (OOBA)

Please check the packing case carefully before opening the box. If there is any damage, please contact the distributor or manufacturer immediately.

WARNING

- The packaging materials must be kept out of the reach of children. They must be disposed of in compliance with local laws and regulations or the hospital policy on waste management.
 - The equipment may be contaminated by microbes during storage, transport and use. Please ensure that the package is undamaged before using, do not use if there is any damage.
-

NOTE

- Keep the packing case and packaging materials for future transportation or storage.
-

3.1.2 Operating Conditions

The operating environment of this veterinary syringe pump must meet the requirements in **A.1.2 Operating Environment**.

The operating environment should also be appropriately protected from noise, vibration, dust, and corrosive, inflammable or explosive substances. If installed inside the equipment case, a sufficient space before and after the equipment case should be ensured to facilitate operation, maintenance and repairing work. There should be a 2" (5 cm) gap around the veterinary syringe pump to ensure that air can circulate freely for a better cooling effect.


When the pump is transferred from one place to another, differences in temperature and humidity can cause condensation to form inside the pump. If this occurs, do not switch the pump to the "ON" state until the condensation has gone.

 **WARNING**


- Please use only when the operating environment meets the requirements specified above. Otherwise, the pump's performance will not match the technical specifications in *A Product Specifications*. Device failure and other unexpected consequences may also result.
-

3.1.3 Mount the Clamp


3.1.3.1 Standard Pole Clamp

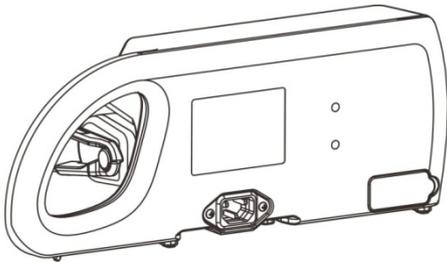

1. Align the mounting holes on the pole clamp with the mounting holes on the back side of the machine, and tighten the screws.

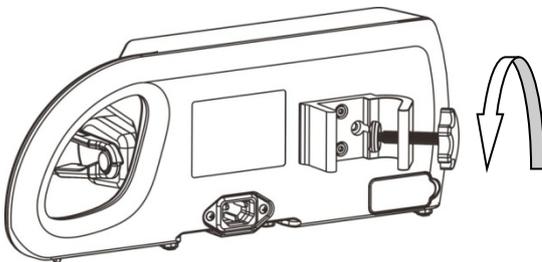


2. Turn counterclockwise to loosen the pole clamp until an IV stand can be inserted in.

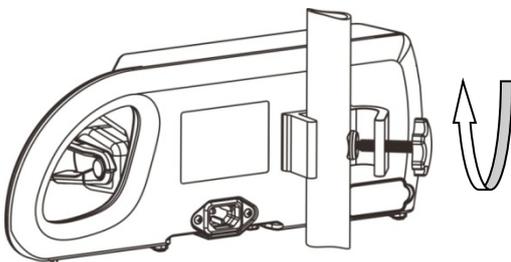

3. Tighten the pole clamp clockwise to firmly fix the device on the IV stand (round vertical bar with diameter size of 15-32mm).

3.1.3.2 Advanced Pole Clamp (Optional)

Align the mounting holes on the pole clamp with the mounting holes on the back side of the machine, and tighten the screws. Press the button of pole clamp, horizontally or vertically adjust pole clamp, the button will pop-up after loosening the pole clamp. Turn the handle, pump can be fixed to cross bar (square cross bar: 10 x 25 mm) or vertical bar (round vertical bar with diameter size of 15-38mm).

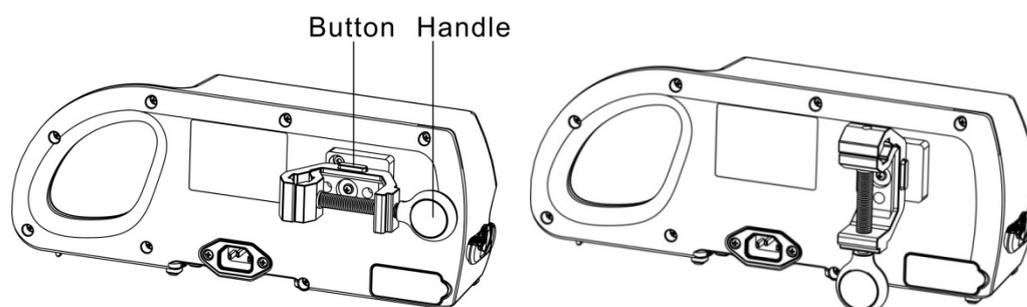

3.1.4 Connect the AC Power Source

The earthing wire in the three-plug connector should be grounded, if there is a doubt whether the AC power system is grounded or not, please adopt the built-in battery and contact an electrical technician at the pet hospital or the company.

WARNING

- **Do not touch the power plug with wet or moist hands! If there is a liquid drug or residue on or around the power socket or plug, the user should completely clean and dry the area before plugging into the power supply, or accidents or injuries may result!**

NOTE

- **Compatible power supply: 100–240 V~, 50/60 Hz.**
- **The AC power cable should be correctly inserted and secured into the socket.**

3.2 Conventional Settings

This chapter only introduces the general settings for the veterinary syringe pump, please refer to other relative chapters for parameters and other feature settings.

3.2.1 Set Language

1. Select [Main Menu]→[Language].
2. Select [Language] from the [Language] according to actual needs.

3.2.2 Adjust Screen Brightness

1. Select [Main Menu]→[Brightness].
2. Select [Brightness]: 1-8. 8 for the brightest setting, and 1 for the darkest setting. When operating on battery power, you can set a low Contrast to save the power of the battery.

3.2.3 Set Date and Time

1. Select [Main Menu]→[System Date and Time].
2. Set [Time] and [Date].

CAUTION

- Please check the system date and time to keep accurate records in the History function.
-

3.2.4 Adjust Volume

1. Select [Main Menu]→[Volume].
2. Select [Volume]: 1-8. 1 for the lowest volume; 8 for the highest volume.

3.3 Restore Factory Default

During operation, you may change some settings in some situations. However, the changes may not be appropriate or correct, especially when animal or syringe brands are changed. Therefore, you should restore the default factory settings during operation according to actual needs, to guarantee that each configuration of the veterinary syringe pump is applicable for animal use. For some default factory settings of this equipment, please refer to **C Default Factory Settings**.

Select [Main Menu]→[System Maintenance]→Input User Maintenance Password→[Restore factory default], and restore the factory default settings as prompted on screen, some parameters will be restored to default values.

FOR YOUR NOTES

4 Basic Operation

4.1 Infusion Flow Chart

Turn pump on	Press  to activate and turn on the pump, the pump will run start-up checks and display the start-up screen;
Install syringe	Please refer to 4.2.2 Insert the Syringe ;
Select brand	Select according to the syringe brand currently being used;
Purge	Refer to 4.2.6 Purge ;
Set infusion parameters	Refer to 4.2.7 Set Infusion Parameters ;
Connect to animal	Connect to the animal;
Start infusion	Press  ;
Finish	Press  ;
Remove syringe	Disconnect from the animal, and remove the syringe in a correct manner;
↓	
Turn pump off	Hold down (>3 s)  , until the Power Off progress bar complete.

4.2 Operational Procedures

4.2.1 Turn on the Pump

After completing the veterinary syringe pump setup, turn on the device as in the following steps:

1. Perform a safety inspection referring to **10.1 Inspection** before turning on the pump.
2. Press , the system will initiate the self-test and the screen will display the **[System Self-test]** interface.
 - ◆ First, the system will give out a sound "di" ——indicating the self-testing of the buzzer to be successful.
 - ◆ Then, the system will give out a sound "du" —— indicating the self-testing of the loudspeaker to be successful.
 - ◆ The color of the alarm indicator lamp will change from red to yellow, turn on and off orderly —— indicating the self-testing of the alarm lamp to be successful.
3. Enter the operation interface after successfully completing the system self-testing, and now you can manually operate the system through the key board.

WARNING


- **Please monitor the self-test process to make sure that the speaker, the alarm light, and the buzzer are all self-tested successfully. Otherwise, please contact the company and do not operate the pump until after maintenance is performed.**
 - **Please contact the company if the veterinary syringe pump is damaged or cannot operate properly, and it cannot be used for animal infusion.**
-

4.2.2 Insert the Syringe


System will inspect whether syringe is installed after completing the self-test:

- If syringe is not installed, enter the syringe **[Installation Guide]** interface.
- If syringe is installed, and only 1 "Commonly used brand": Enter the **[Syringe selection]** interface if the switch of "Brand" is "On"; Enter the infusion parameters setting interface if the switch of "Brand" is "Off".
- If syringe is installed, and the number of "Commonly used brand" is ≥ 2 , enter the **[Syringe selection]** interface.
- If the syringe is not required to install, press  to skip the step.


Install syringe according to the following method:

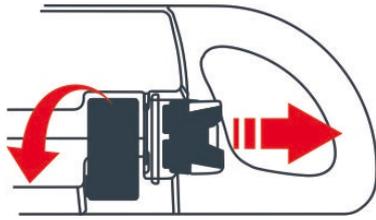

1. Open the syringe fixation clamp, squeeze the handle to open the clip, and then move the slider to the appropriate position.

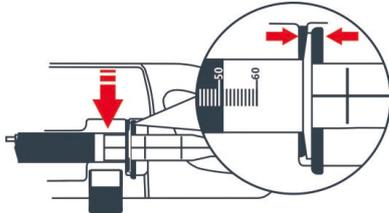

2. Align the syringe flange with the slot, then insert it into the slot, and clamp the syringe.
(Tip: Flange location)

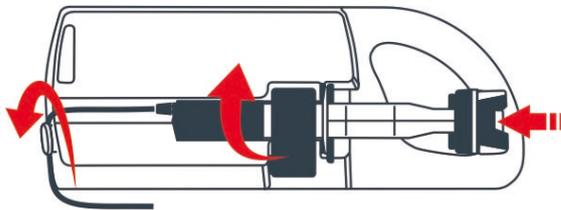

3. Align the slider to and snugly against the syringe plunger, making the clip clamps firm to the thumb rest. Close the syringe clamp gently.

WARNING

- The flange of the syringe should be firmly inserted into the slot, and not jutting on the outside of the flange plate.
- Before using this veterinary syringe pump, the veterinary syringe pump, syringe and other accessories should be installed correctly.
- Before using this veterinary syringe pump, the brand and specifications of the syringe used must be confirmed. The brand of veterinary syringe pump should be calibrated on the equipment. If there are no settings for the syringe used, the rate and the alarms may not be accurate.
- Only one option in “Commonly used syringes” and the switch of “Brand” is “Off”, if the installed syringe is not the selected brand in “Commonly used syringes”, its accuracy cannot be guaranteed.

4.2.3 Change the Syringe

Follow the steps below to change the syringe:

1. To prevent animal injury, before changing the syringe during infusion, press  to stop the pump.
2. Open the syringe fixation clamp, squeeze the handle to open the clip, move the slider to the appropriate position, and take out the installed syringe.
3. Please refer to **4.2.2 Insert the Syringe** to reinstall the syringe.

4.2.4 Select Syringe Brands

On the [**Syringe selection**] screen, press   to select the syringe brand and specifications of the syringe currently being used, and press  to confirm the syringe brand.

WARNING



- A new syringe brand should be calibrated when used for the first time.

CAUTION

- Please confirm that the current selected brand is the same as the brand actually used, or its accuracy cannot be guaranteed.

4.2.5 Infusion Mode

Press  to exit [**Main Menu**], and enter the [**Rate Mode Setting/Running**] interface.

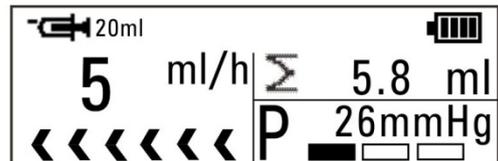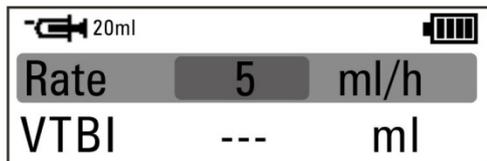

Mode	Parameters	Parameter Range
Rate Mode	Rate	5ml syringe: 0.1-150ml/h 10ml syringe: 0.1-300ml/h 20ml syringe: 0.1-600ml/h 30ml syringe: 0.1-900ml/h 50ml/60ml syringe: 0.1-1500ml/h
	VTBI	0.1-9999ml

4.2.6 Purge

During infusion, the user should prevent air bubbles from entering the blood with the liquid drug, which may form an aeroembolism and put animals in serious danger. Therefore, air bubbles in the syringe and extension cannula should be eliminated

before the infusion. On infusion parameters setting interface, press  to enter [Purge] prompt screen. Hold down  to enter [Purge] running screen, release  after the air bubbles are purged.

WARNING

- **During the purge, please disconnect the pump from the animal. Otherwise, the animal will be in serious danger!**

NOTE

- **Purge rate can not be changed.**

4.2.7 Set Infusion Parameters

Under rate mode, users can set infusion parameters by pressing , , , , and .

4.2.8 Infusion

When ready, connect the extension cannula to the animal. Press  to start the infusion, and the screen will display the running icon, the arrows will move from right to left, and the running speed will increase, which will indicate that the rate will also increase.

WARNING

- **Users should regularly monitor the connection between the syringe, extension cannula, pump and animal, and infuse according to the method mentioned in the manual.**

NOTE

- When in running status, if there is no operation in other interface over 2 minutes, it will return to the running screen automatically.
-

4.2.9 BOLUS

In run screen in rate mode, press  to enter the [Bolus] settings screen. Follow the ways to start the bolus:

- Manual Bolus: Set bolus parameters, and then press and hold  to start a manual bolus. The system returns to the original rate when  is released, or  is hold for 10 seconds.

NOTE

- If the maximum rate of the specification of syringe <800ml/h, the default bolus rate is the maximum rate of the specification of syringe.
 - If the maximum rate of the specification of syringe >800ml/h: current rate <800ml/h, and the bolus rate is 800ml/h ; if current rate >800ml/h, the default bolus rate is the maximum rate of the specification of syringe.
 - If no operation is performed within 2 minutes, the veterinary syringe pump will automatically exit the Bolus Settings screen and the procedure must be repeated.
 - [VTBI Near Done] and [Syringe Nearly Empty] alarm will not be triggered during bolus.
 - Occlusion pressure will automatically switch to "High" level when bolus, animal's clinical condition and working condition of the veterinary syringe pump must be monitored carefully.
-

4.2.10 Change the Rate during Operation

In run screen in rate mode, press ,  or  to change the value of the

[Rate] into the adjustable state, thus to set the expected rate, press  or  again for confirmation, then start to infuse under the new set rate.

4.2.11 Complete

If [VTBI] is not set during the infusion, when the time in which the remaining liquid needs to be reached [Near End] time, the [Syringe Nearly Empty] alarm will be triggered, and this alarm cannot be cancelled until the syringe is empty. Set [Near End] time, please refer to **5.5 Time Near End**.

When the remaining infusion time is close to the [Near End] time set by the users, the [VTBI Near Done] alarm will be triggered. If no action has been taken, the alarm will not be cancelled automatically until the infusion is completed, and then switch to [VTBI Done] alarm.

After KVO mode runs for 30 minutes, the [KVO Finish] alarm will be triggered. Set KVO rate, please refer to **5.1 KVO**.

4.2.12 Standby

Under non-operation status, tap (<3 s)  to enter [Standby] interface, default display the previous standby time, press  to modify standby time (range is 00:01-99:59 hh:mm), press  for confirmation after modifying. The pump cannot be put in standby mode if there is a high-level alarm.

When the standby state is ended, the title bar will display [Standby Time Expired], press  to confirm and quit, until the screen before standby appears. Press  to remain in standby.

4.2.13 Turn off the Pump

Follow the steps below to turn off the veterinary syringe pump:

1. Disconnect from the animal;
2. Hold down (>3 s) , until the Turn Off progress bar complete, and the power will turn off.

NOTE

-
- **The current operating data and saved data will always be autosaved when the device is powered off properly. The effective storage duration of the data is equal to the service life of the device.**
-

FOR YOUR NOTES

5 Setting Parameters

5.1 KVO

KVO (Keep Vein Open) means to keep the vein open, during which the veterinary syringe pump continues infusion at a very low rate after finishing the infusion in order to prevent blood backflow or vascular occlusion.

Select **[Main Menu]**→**[KVO rate]**: 0.5ml/h is unadjustable.

5.2 Occlusion Pressure (unit)

Occlusion pressure is adjustable, which can meet the requirements for occlusion pressure of different animals during infusion. Pressure in the infusion tube can be measured by the built-in pressure sensor, pressure can be calculated by the internal CPU, which is compared with the preset occlusion alarm threshold. **[Occlusion]** alarm will be triggered if pressure exceeds the threshold.

5.2.1 Set Occlusion Pressure

1. Select **[Main Menu]**→**[Occl. pressure]**.
2. Select **[Occl. pressure]**: Occlusion pressure Degree 3, lowest at 300±125mmHg, and highest at 900±180mmHg. Occlusion pressure should be selected according to actual needs.

CAUTION

- If the animal experiences discomfort at a higher occlusion pressure, monitor the animal's physical conditions under the higher occlusion pressure closely, and take measures instantly if any abnormal condition occurs.
-

5.2.2 Set the Pressure Unit

1. Select **[Main Menu]**→**[Pressure Unit]**.
2. Select **[Pressure Unit]**: The 4 various forms of pressure units, mmHg, kPa, bar and psi are converted automatically, and can be selected according to actual needs.

CAUTION

- Carefully confirm the edit when changing the current pressure unit.
-

5.2.3 Dynamic Pressure Scanning (DPS)

During the infusion, the bottom-right corner of the Run screen demonstrates real-time pressure changes of the animal in order to find the cannula occlusion at an earlier time and to prevent the occurrence of further complications.

The pressure icon  on the screen indicates the condition of the current pressure:

- 1 solid area indicates low occlusion pressure
- 2 solid areas indicate medium occlusion pressure
- 3 solid areas indicate high occlusion pressure

5.2.4 Automatic Pressure Release Function (Anti-Bolus)

When occlusion occurs, infusion will stop and the [**Occlusion**] alarm will be triggered. After the alarm is triggered, the motor is reversed, and the cannula pressure is then released. This prevents an additional aggressive dose to the animal after the occlusion is eliminated.

5.3 Key Lock Function

When locked, an  icon in the upper-right corner of the screen emerges. The following are two ways for automatic locking and manual locking:

- Automatic Locking:
 1. Select [**Main Menu**]→[**Auto-lock**].
 2. Select [**Auto-lock**]: Off, 1-5min. After a specific time is set during the running state, and if there is no operation or high-level alarm within the set Auto-lock time, the key board will be auto-locked. [**Off**] indicates closing automatic locking function.
- Manual Locking: In the running interface, under the unlocking condition, press and hold (>3 seconds)  to lock the key board.

Note: If unlocking is needed, press and hold (>3 seconds)  to unlock, it is automatically locked during the high-level alarm.

5.4 Reminder Function

1. Select **[Main Menu]**→**[Reminder]**.
2. Select **[Reminder]**: Off, 1-5min. After a specific time is set, the syringes are to be inserted. No operations are performed by the pump within the set time (including operations on the keyboard, the slider and the pull handle), and the **[Reminder]** alarm will then alert the user to proceed to the next step. **[Off]** indicates closing the function.

5.5 Time Near End

1. Select **[Main Menu]**→**[Near End]**.
2. Select **[Near End]**: Off, 1-30min (when <10min, the stair-step is 1min, and when ≥10min, the stair-step is 5min). After a specific time set, when the remaining infusion time is close to the **[Near End]** set by the users, **[VTBI Near Done]** alarm will be triggered. **[Off]** indicates closing the function.

5.6 Common Syringe Brands

There are multiple commonly used syringe brands installed inside the veterinary syringe pump, making it convenient for the user to select from. Only 5 ml, 10 ml, 20 ml, 30 ml and 50 ml/60ml syringes that comply with national standards should be used with this veterinary syringe pump. For specific syringe brands, please refer to actual syringes as the standard.

1. Select **[Main Menu]**→**[Commonly used brand]**.
2. Select in **[Commonly used brand]** according to actual needs.

Built-in Syringes List

No.	Brand and Model	5ml	10 ml	20 ml	30 ml	50 ml	60 ml
1	B.Braun OPS			√		√	
2	BD Luer-Lok Tip	√	√	√	√		√
3	Terumo SS	√	√	√	√	√	
4	Dove	√	√	√	√	√	
5	Jierui	√	√	√	√	√	
6	QiaoPai	√	√	√	√	√	
7	SuYun	√	√	√	√	√	
8	ShanChuan SCZ	√	√	√	√	√	
9	HuaFu HF009	√	√	√	√	√	

5.7 Bed No. Settings

1. Select [**Main Menu**]→[**System Maintenance**]→Input User Maintenance Password →[**Bed No.**].
2. Select [**Bed No.**]: 1-999. [---] indicates invalid values. Beds can be differentiated through setting bed number.

5.8 View Department

The veterinary syringe pump are net-connected with BeneFusion CS5 Infusion Supervision System through the approach of wireless networking, and the system contains department information, the system will automatically distribute department information to all veterinary syringe pumps when pumps are on.

Select [**Main Menu**]→[**System Maintenance**]→Input User Maintenance Password→[**Department**] to view the department information.

6 Other Functions

6.1 Nurse Call

Select **[Main Menu]**→**[System Maintenance]**→Input User Maintenance Password
→**[Nurse Call]**, and set in the open menu:

- Switch

On: Indicates the opening of the nurse call function.

Off: Indicates the closing of the nurse call function.

- Signal type

1. Cont.

Indicates that the output nurse call signal type is the same as that of the alarm existence time, i.e., from the occurrence of the alarm to the end of it.

2. Pulse

Indicates the output nurse call signal is a pulse signal with the type of 1 second.

When several alarms exist at the same time, only one pulse signal can be outputted.

If the current alarm is not removed and another alarm occurs, then one additional pulse signal is outputted.

- Trigger type

1. Normally Closed: Select when the pet hospital call system is set as **[NC]**.

2. Normally Open: Select when the pet hospital call system is set as **[NO]**.

- Alarm level: Three options: **[High]**, **[Medium]** and **[Low]**. The system sends nurse call signals according to the alarm at the selected alarm level or above.

WARNING

- **Non-veterinarians are forbidden to modify the nurse call setting.**
 - **The nurse call function must be used in conjunction with a special cable.**
-

NOTE

- **Veterinarians should not consider the nurse call function as the main alarm notice approach, and rather combine the sound and visual alarms of the veterinary syringe pump and the performances and symptoms of the animal in order to judge the animal's conditions and take further attention as needed.**
-
-

6.2 Wireless Networking (Optional)

The veterinary syringe pump can be configured with the wireless modules, and be net-connected with BeneFusion CS5 Infusion Supervision System through the approach of wireless networking. Through the network:

1. The veterinary syringe pump sends real-time infusion parameters, alarm information, prompt information, and bed number and so on to BeneFusion CS5 Infusion Supervision System.
2. BeneFusion CS5 Infusion Supervision System and the veterinary syringe pump can display synchronously. For detailed descriptions, please refer to the instructions of BeneFusion CS5 Infusion Supervision System (hereinafter called CIMS).

Normal communication of the pump and CIMS depends on whether the network connection is successful, operators are unable to observe the operation status of the pump in real time when the communication is interrupted. After the network connection settings of the pump and CIMS are modified, operators shall reset the network connection as required in the manual to ensure the communication of the pump and CIMS are restored.

When using wireless modules to connect to the Internet while using the veterinary syringe pump, the wireless icon on the upper-right corner indicates the working condition of the wireless modules:

-  Configured wireless module and connected successfully
- No icons No wireless modules configured or no connection

NOTE

-
- **Wireless security transmission distance is no more than 50 meters.**
 - **2.4 GHz Wi-Fi frequency range, WEP and WPA/WPA2 security modes and 802.11b/g/n wireless standard are supported.**
 - **The settings of the wireless network must be conducted by technicians approved by the company or maintenance staff designated by the company.**
-

6.3 Data Export

To export the data in the veterinary syringe pump, please refer to the following steps:

1. Log on PC tools, and connect the PC to the veterinary syringe pump;
2. When the veterinary syringe pump is in working communication with the PC, the PC automatically reads all the data in the pump;
3. Select [**History Record**] in PC tools, and export data.

6.4 WLAN Setting

The pump can be net-connected through built-in Wi-Fi module.

1. Select [**Main Menu**]→[**WLAN Setting**], then select [**On**] to enable Wi-Fi function.
2. Select [**Advanced Settings**], there are two ways to distribute IP address:
 - DHCP: Check the checkbox to activate DHCP, IP address, subnet mask and gateway can not be modified, automatically obtain an IP address.
 - Manually: Uncheck the DHCP checkbox, enter IP address, subnet mask and gateway.
3. Available networks shall be displayed:
 - If password is required for to-be-connected network, please enter the password.
 - If password is not required for to-be-connected network, you can connect the network directly.

FOR YOUR NOTES

7 Alarms

The alarm is used in order to alert the veterinarian by means of sound and light when abnormal situations occur during the infusion procedure which can lead to infusion changes or when the infusion of the animal cannot continue due to the unexpected breakdown or pause/delay of the veterinary syringe pump.

WARNING

- It is potentially hazardous to use the same or similar equipment with different alarm presets within the same area.
-

7.1 Alarm Level

According to the severity scale of the alarm, the alarms of the veterinary syringe pump can be classified to be high-level alarms, mid-level alarms and low-level alarms.

7.2 Alarm Types

When an alarm is triggered, the veterinary syringe pump will use the following visual and audible methods to alert the user:

- Visible alarms
- Audible alarm
- Alarm Information

Among the visible alarms and audible alarms, the alarm information will distinguish alarm levels in different ways.

Alarm level	Color of alarm light	Audible alarm frequency	Flashing light frequency	Light/no-light ratio
High-level alarms	Red	10 seconds	2.0±0.6Hz	20%~60%
Mid-level alarms	Yellow	15 seconds	0.6±0.2Hz	20%~60%
Low-level alarms	Yellow	20 seconds	Steady	100%

7.2.1 Multi-level Alarm Rules

When several alarms occur simultaneously, the alarms proceed according to the following rules:

- When several alarms at different levels occur, the visible alarms and audible alarms are consistent with the highest-level alarms.
- When several alarms at different levels occur, only the highest-level alarm is displayed, and after it is cancelled, the lower-level alarm will then be displayed.
- When several alarms at the same level occur, the alarm information will be demonstrated in an alternate manner.
- When [VTBI Near Done] alarm and [Syringe Nearly Empty] alarm occur simultaneously, only [VTBI Near Done] alarm is triggered.

The title bar of the veterinary syringe pump screen will display the corresponding alarm information during the alarm blast, to see more details in ***D Alarm Information***.

NOTE

-
- The [No Communication] alarm of the pump and BeneFusion CS5 Infusion Supervision System are delayed for 3 minutes, while other alarms are delayed for less than 5 seconds.
-

7.3 Alarm Handling Rules

Under normal working conditions, when an alarm occurs, all the alarm types of the veterinary syringe pump will alert according to their respective alarm levels. In addition, the user can pause the alarm sound according to demands.

- For high level (except battery empty) and medium level alarms, press  to pause alarm sound for 2 minutes, no alarm sound is made in any case. When the alarm pause time expires, the alarm tone will sound. Press  to cancel high level alarms (except battery empty and system error).
- For low level alarms, press , no alarm sound, alarm information and alarm light, until it is triggered next time.

NOTE

-
- [Battery Empty] alarm sound is unable to be paused.
-

7.4 Alarm Countermeasures

WARNING

- **When an alarm is triggered, the animal's condition should be checked firstly and operation should only be allowed to proceed after the reason for the triggering of the alarm is ruled out.**
-

When an alarm is triggered, please follow these steps and take appropriate action:

1. Check the animal;
2. Check the alarm type and the parameter which triggered the alarm;
3. Determine the reason for the alarm;
4. Eliminate the reason for the alarm;
5. Check whether the alarm has cleared.

NOTE

- Please refer to *D Alarm Information* for specific handling procedures for each alarm.
 - The operator position shall be the normal operating position of the veterinary syringe pump (0.5m). Otherwise, operators might identify alarm mistakenly.
-

FOR YOUR NOTES

8 Battery

WARNING

- **The battery can not be disassembled. The battery should be changed by maintenance staff designated by the company only. Inserting fuel cell or changing battery by personnel who has not received suitable training may cause such danger as overtemperature, fire or explode.**
-

The veterinary syringe pump is configured with rechargeable Lithium ion batteries to ensure that the veterinary syringe pump can be used normally under the condition of the animal's migration within the pet hospital or during the circumstance of a power failure. When the veterinary syringe pump switches to the AC power, the battery can be charged regardless of whether the veterinary syringe pump is on or off. The battery is chargeable only within the veterinary syringe pump. During charge, the battery icon in the upper-right corner of the screen floats left and right. If the battery icon stops floating and is completely filled, it indicates that the battery is fully charged. Under the condition of a sudden power failure, the system will automatically use the battery to provide power as a backup.

The battery icon on the screen indicates the condition of the battery:



The battery jar of the veterinary syringe pump is installed with batteries, and the white fill area indicates the quantity of electricity.



Low battery electric quantity indicates that charging is needed.



When the battery is empty, charging is needed immediately.

The power supply by the battery can only be sustained for a limited period of time. The **[Battery Empty]** alarm will be triggered when the battery voltage is too low, and red alarm light will flash. The alarm will continue within the remaining time of the battery's electric quantity and cannot be paused. Now, the veterinary syringe pump should be switched on to AC power for charging.

8.1 Battery Performance Optimization

When the battery is used for the first time, at least two complete optimizing cycles should be ensured. A complete optimizing cycle contains the following: Charging incessantly, and then discharging until the power of the veterinary syringe pump runs out. During usage, regularly optimizing the battery performance will extend its lifespan. It is suggested that the battery should be optimized when in use or in storage for three months, or when the running time of the battery is significantly shortened.

Please follow the steps below during optimization:

1. Disconnect the pump from the animal and stop the infusion.
2. Switch the veterinary syringe pump on the AC power and charge the battery incessantly for over 10 hours.
3. Disconnect the AC power and use the battery to charge the veterinary syringe pump until the veterinary syringe pump is closed.
4. Switch the veterinary syringe pump over to AC power again and charge the battery incessantly for over 10 hours.
5. The battery optimization is now complete.

8.2 Check the Battery

The performance of the battery may decrease over time. Please follow the steps below when checking the battery:

1. Disconnect the pump from the animal and stop the infusion.
2. Switch the veterinary syringe pump on the AC power and charge the battery incessantly for over 10 hours.
3. Disconnect the AC power and use the battery to charge the veterinary syringe pump until the veterinary syringe pump is closed.
4. The length of the battery's lifetime reflects the performance of the battery.

Note: If the length of the battery's lifetime is obviously shorter than that claimed in the specifications, please consider changing the battery or contact us.

NOTE

- The lifespan of the battery depends on how frequently it is used and on how long it has been used, battery capacity decreases with increase in charging and discharging times. If the maintenance and storage of the battery is appropriate, the lifespan of the Lithium ion battery is no less than 300 times of full charging and discharging. If the use of battery is improper, its lifespan shall be shortened or in failure status. We recommend replacing the lithium battery every 3 years.
 - Please connect to the AC power source if [Battery Empty] alarm is triggered. To prevent battery not used for a long time or in battery empty status, if battery is not charging more than two months after battery is empty, battery will be in failure status. Do not charge the failure battery, and replace the failure battery.
 - If battery will not be used for a long time, we recommend keeping the battery in a fully charged state and charging the battery every two months for lifespan guarantee. Please replace the battery if the length of its lifetime is obviously shortened during optimization.
 - The length of the battery's lifetime depends on the device configuration and operation, for example: Under the condition of the power supply by the battery, frequent infusion at a high rate will also shorten the length of the battery's lifetime.
-

8.3 Battery Recycling

If there is any obvious damage to the battery or to the battery capacity exhausts, it should be replaced and recycled appropriately. Please follow the applicable laws on recycling.

WARNING

- The battery must not be disassembled, burned or short-circuited. Burning, exploding or leaking batteries can cause animal injury.
-

FOR YOUR NOTES

9 Preservation and Sanitation

The pump must be cleaned or disinfected using the materials and methods listed in this section. The manufacturer will not be responsible for any damage or accident caused by cleaning and disinfection using other materials and methods.

The manufacturer shall not be held responsible for the efficacy of the following chemicals or methods for infection control. Please contact your pet hospital's infection prevention department or epidemiology specialists for advice on infection control practices.

9.1 Description

Please make sure that your device and other fittings are clean without dust. In order to prevent any damage to the device, please abide by the following rules:

- Dilute all cleaning agents and disinfectants in accordance with the manufacturer's instructions, or use as low a concentration as possible.
- Do not immerse or submerge the device in liquid.
- Do not pour liquid on the device or accessories.
- Avoid liquid from entering the pump body.
- Do not use abrasive materials (such as steel wool or silver polishes), or any strong solvent (such as acetone or any detergent containing acetone).

WARNING

- **Turn off the pump and disconnect the AC power source line from the socket before cleaning. Do not clean and disinfect the device, export history record and perform other operations when animals are using the pump.**
-
-

9.2 Cleaning

The pump should be cleaned regularly. If operating in dirty or sandy areas, cleaning should be more frequent. Before cleaning, please consult or refer to the pet hospital's specific regulations concerning medical device cleaning.

The recommended detergents include: Hydrogen peroxide (3%).

To clean your equipment, follow these rules:

1. Turn off the pump and disconnect the AC power source line.
2. Wipe the display screen after soft cotton balls absorb an appropriate amount of detergent.
3. Use a piece of soft cloth which absorbs a modest amount of cleaning agent to wipe the surface of the device.
4. When necessary, use a piece of cloth to wipe off any excess cleaning agents.
5. Place the pump in a cool and ventilated environment to dry.

9.3 Disinfection

The operation of disinfection may cause certain damage to the veterinary syringe pump. You are recommended to disinfect only when it is necessary in your desired maintenance plan. Clean the equipment before disinfection.

The recommended disinfectants include: glutaraldehyde-type 2% liquid disinfectant.

CAUTION

- **Never use EtO or formaldehyde for disinfection.**
 - **Do not conduct high pressure or high temperature disinfection for the veterinary syringe pump and its accessories.**
-

10 Maintenance

WARNING

- **The pet hospital or medical facility using this veterinary syringe pump must set up a comprehensive maintenance plan. Failure to do so may result in equipment failure or other unexpected consequences, and may even jeopardize animal safety.**
 - **All safety inspections or maintenance work involving the disassembly of the device must be conducted by professional maintenance personnel. Actions by unqualified persons may result in device failure and may even jeopardize animal safety.**
 - **Please contact the company immediately if you encounter problems with the device.**
-

10.1 Inspection

The veterinary syringe pump must be given a thorough inspection before use, after 6-12 months of continuous use, and after maintenance or upgrades, to ensure that it is operated and functioned normally.

The inspection criteria are:

- The environment and power supply meet requirements.
- The equipment and accessories have no mechanical damage;
- The power cord is not damaged and has sufficient electrical insulation
- Accessories used with the pump are correct
- The alarm system functions correctly.
- Battery performance.
- Self-checking and pump functions are normal.

If there are any forms of damage or abnormal circumstances, do not use the veterinary syringe pump and contact the company immediately.

10.2 Maintenance Plan

The following tasks must be conducted by professional maintenance personnel approved by the company. Please contact the company if the following

maintenances are needed. Must clean and disinfect the device before the test or maintenance.

Inspection/Maintenance Items	Frequency
Perform a safety inspection according to the IEC60601-1 standard.	Once every two years. Perform after the board is changed or the veterinary syringe pump is accidentally dropped.
Preventive maintenance (refers to the Maintenance Manual for pressure calibration, sensor calibration, and pump inspection).	Once every two years, or when you suspect the occlusion alarm is abnormal, the flow volume is inaccurate, or the syringe is incorrectly identified.

10.3 View Information

Select **[Main Menu]**→ **[Version Information]**. In the **[Version Information]** interface, you can view the information of the version of the veterinary syringe pump system and other versions.

10.4 Syringe Calibration


The syringe in the veterinary syringe pump needs no daily calibration. But calibration is required when the veterinary syringe pump is used for the first time, is replaced with a new syringe brand or when you suspect the deviation of the flow volume is much larger.

Prepare the following materials before the calibration:

- Syringe: 5 standard syringes of the corresponding full dimensions of 5 ml, 10 ml, 20 ml, 30 ml, and 50 ml/60 ml, without liquid.

The steps for calibration are as follows:

1. Pull the syringes to the full dimensions and install them on the veterinary syringe pump. "L" in the fig. below refers to the full dimension of the 50 ml syringe.
2. Open the **[Syringe Calibration]** interface: Select **[Main Menu]**→**[System Maintenance]**→Input User Maintenance Password→**[Syringe Calibration]**.
3. Select **[Brand]** and **[Size]** of the current syringe in the **[Syringe Calibration]** interface.
4. Press  and the veterinary syringe pump starts automatic calibration.
5. The screen prompts **[Calib done]** after the calibration is successful.
6. Press  to exit the current interface.

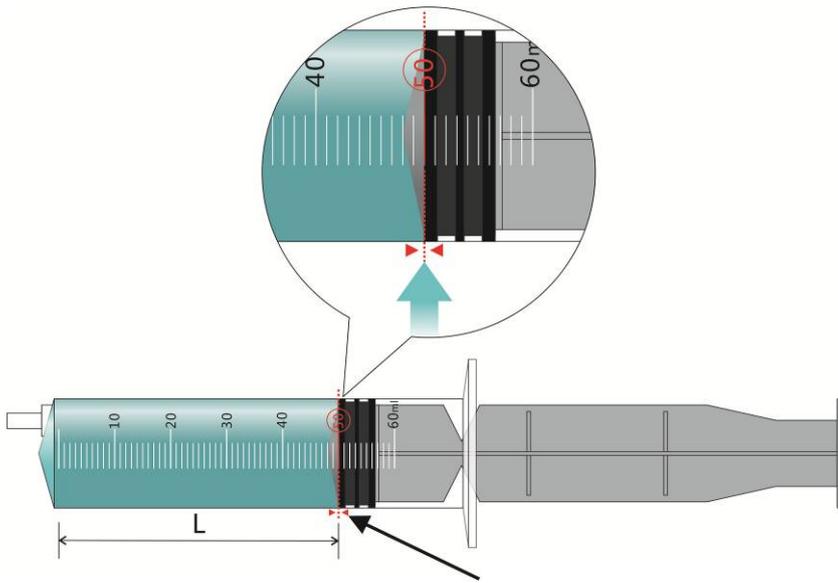

Please align the front end of the plunger rod's sealing plug with the full dimension mark

NOTE

-
- The full dimension of the 50/60 ml syringe is the distance from 0~50 ml.
-

10.5 Safe Disposal and Recycling

Please contact the company for related information about safe disposal and recycling.

FOR YOUR NOTES

11 Accessories

WARNING

- Use the accessories specified in this chapter only. Other accessories may cause damage to this veterinary syringe pump, or cannot reach the specification in this manual.
 - Please do not replace an accessory if its package or itself is damaged.
-
-

Materials	PN
Power cord (Select PN according to sales area)	009-002755-00
	009-002756-00
	009-003358-00
	009-002757-00
	009-003651-00
	009-002758-00
Standard pole clamp	115-031551-00
Advanced pole clamp	115-031552-00
Nurse call cable	115-034140-00
RS-232 communication cable	115-034142-00
DC input cable	115-034144-00
Pet bracket	115-033273-00

NOTE

- This Operator's Manual describes the most complete functional configuration of the system. The device you are using may not have some of the settings or functions described herein.
-

FOR YOUR NOTES

A Product Specifications

A.1 Safety Specifications

A.1.1 Product Classification

Classifications of this veterinary syringe pump according to the IEC60601-1 standard are as follows:

Safety	
Components	Host
IEC protection class	I
Protection against electric shock	CF Protected against defibrillation
Liquid ingress protection	IP34
Explosion protection level	Unsuitable
Operating mode	Continuous
Mobile level	Portable

NOTE:

- I: Type I devices
- CF: Class CF applied parts, can be directly used in the heart.
- IP34: Protected against solid foreign objects with a diameter no less than 2.5mm and protected against spraying water.
- Unsuitable: The device is unsuitable to be used in environments containing air mixed with flammable anesthetic gas, oxygen or nitrous oxide.
- Portable devices: Can be moved from one place to another by one or more persons or by other means when the devices are in use or being used.
- Portable veterinary syringe pump: Used to control the infusion of animals and are devices which can be carried by the animals continuously.

A.1.2 Operating Environment

Work environment	
Temperature	5 - 40°C
Relative Humidity	15 - 95%, non-condensing
Atmospheric pressure	57 - 106 kPa
Storage environment	
Temperature	-20-60 °C
Relative Humidity	10–95%, non-condensing
Atmospheric pressure	50–106 kPa
Storage conditions	Corrosive-free and ventilated indoors
AC Power Supply	
Voltage	100 - 240 V~
Frequency	50/60 Hz
Current	0.40-0.14A
Fuse	Low interrupting rating, T5AL/250V
External DC power supply	
Voltage	DC 10V-16V
Current	2.00-1.25A

A.2 Physical Specifications

Components	Weight	Size	Remark
Host	Less than 1.8 kg (Without pole clamp)	Less than 336 x 132 x 110 (mm) (length × width × depth) (Without pole clamp)	One battery included

A.3 Hardware Specifications

A.3.1 Display

Display	
Type	Monochrome LCD
Size (diagonal)	2.5 inches
Differentiation	132 x 32 pixels

A.3.2 Battery

Internal battery	
No. of batteries	1 (standard) or 2 (optional)
Battery type	Li-Ion ion battery
Shutdown delay	At least 30 mins (new battery, after the first low battery alarm)
Rated battery voltage	7.4 VDC (1 battery) or 5200 mAh (2 batteries)
Battery capacity	2600 mAh
Power supply time	Continuously operate at a rate of 5 ml/h, disable Wi-Fi function, discharge for at least 6h (1 battery) or 12h (2 batteries) using a fully charged new battery.
Charging time	When the pump is off, the charging time is not longer than 5h (1 battery) or 10h (2 batteries).

A.3.3 Host LED

Host LED	
Alarm light	1 (two colors: red and yellow)
AC/DC indicator light	1 (green)
Battery indicator light	1 (green)

A.3.4 Auditory Indicator

Speaker	Produce an alarm, the sound pressure is 59-75 dB(A) and key beep; Support multi-level volume functions; The alarm sound meets the requirements of the IEC60601-1-8 standard.
---------	--

A.3.5 External Ports

Ports	
AC power supply port	1 AC power supply port
Multifunction interface	1 multifunction interface, which combines the following interface functions: <ul style="list-style-type: none"> ■ DC power input interface ■ RS232 interface ■ Nurse call interface

A.3.6 Signal Output Interface

Nurse call signal output	
Driving mode	High level
Electric specification	≤60W, ≤2A, ≤36VDC, ≤25VAC
Isolation voltage	>1500 VAC
Action mode	Normally open or normally closed (optional)

A.4 Wi-Fi Specifications

Protocol	IEEE 802.11b/g/n
Modulation mode	CCK and OFDM
Operating frequency	2412~2484MHz
Channel spacing	5MHz
Data rate	IEEE 802.11b: 1 - 11 Mbps IEEE 802.11g: 6 - 54 Mbps IEEE 802.11n: 6.5 - 150 Mbps
Transfer power	802.11b: +18dBm(Max) 802.11g: +16dBm(Max) 802.11n: +15dBm(Max)
Operating mode	STA/AP/STA+AP
Data security	WEP/WPA-PSK/WPA2-PSK

A.5 Specifications

Parameters	Specifications
Syringe standard	Syringe used in conjunction with veterinary syringe pump should meet the requirements of ISO 7886-1: Sterile hypodermic syringes for single use.
Compatible syringe sizes (ml)	5 ml, 10 ml, 20 ml, 30 ml, 50 ml /60 ml
Rate range	<ul style="list-style-type: none"> ■ 5ml syringe: 0.1-150ml/h ■ 10ml syringe: 0.1-300ml/h ■ 20ml syringe: 0.1-600ml/h ■ 30ml syringe: 0.1-900ml/h ■ 50ml/60ml syringe: 0.1-1500ml/h
Increment of rate	<ul style="list-style-type: none"> ■ 0.1ml/h-999.9ml/h, the increment is 0.1ml/h ■ 1000 ml/h -1500 ml/h, the increment is 1ml/h
Bolus rate range	<ul style="list-style-type: none"> ■ 5ml syringe: 0.1-150ml/h ■ 10ml syringe: 0.1-300ml/h ■ 20ml syringe: 0.1-600ml/h ■ 30ml syringe: 0.1-900ml/h ■ 50ml/60ml syringe: 0.1-1500ml/h <p>Note: Bolus accuracy is not declared.</p>
Purge rate range	<ul style="list-style-type: none"> ■ 5ml syringe: 150ml/h ■ 10ml syringe: 300ml/h ■ 20ml syringe: 600ml/h ■ 30ml/50ml/60ml syringe:800ml/h
VTBI range	0.1-9999 ml, the increment is 0.1 ml
Volume range	0.1-9999 ml, the increment is 0.1 ml
Standby time range	00:01-99:59 hh:mm
Mode selection	Rate Mode
KVO rate	0.5ml/h
Anti-bolus	On, Off
Occl. pressure	Low, Medium and High, respectively are 300±125mmHg (40.0±16.7kPa), 525±125mmHg (70.0±16.7kPa), 900±180mmHg (120.0±24.0kPa).Maximum occlusion pressure is about 1300mmHg.
Pressure unit	mmHg, kPa, bar and psi
Auto-lock	Off, 1 - 5 min, step for 1min
Reminder	Off, 1 - 5 min, step for 1min

Near end	Off, 1- 30 min when the time is <10min, step for 1min, and step for 5 min when the time is ≥10 min
Bed No.	1-999
Volume	1 - 8
Brightness	1 - 8
System date and time	Time: __:__:__
	Date: ____-__-__
System language	You can select language according to actual needs
Nurse call	On, Off
Infusion accuracy	Infusion accuracy of the whole rate $\leq \pm 2\%$ or $\pm 0.005\text{ml/h}$, whichever is larger.
Mechanical accuracy	Mechanical accuracy error $\leq \pm 1\%$
Alarm Information	Occlusion, Battery Empty, VTBI Done, KVO Finish, Syringe Empty, Syringe Disengaged, System Error, System abnormal, Reminder, Battery Low, Syringe Nearly Empty, VTBI Near Done, Standby Time Expired, AC Power Disconnection, Syringe not inserted and No communication
Status indicators	Stop, infusion, bolus, KVO, standby, alarm and purge
Dose of single fault	About 3.5ml

A.6 A Reference Table Showing Occlusion Alarm

Delay and Possible Dose

Occlusion pressure (Level)	Rate(ml/h)	Time of occlusion alarm (hh:mm:ss)	Bolus(ml)
Low	0.1	03:50:27	0.021
	1	00:19:26	0.020
	5	00:04:33	0.013
Medium	1	00:35:50	0.014
	5	00:08:46	0.016
High	0.1	13:45:04	0.019
	1	01:21:03	0.015
	5	00:11:21	0.018

NOTE


-
- **Test conditions:**
 - ✓ FLUKE IDA4 PLUS tester
 - ✓ Syringe brand: Dove
 - ✓ Syringe specification: 20ml
 - ✓ Test temperature: 20±2°C
 - ✓ Extension tube length: 1 meter
 - **Occlusion alarm pressure, alarm delays and bolus volume may vary depending on test conditions, temperature and tube length.**
 - **The above data are only typical values under normal test conditions. The actual data may vary as test conditions change. Please refer to the test data for the product you have purchased. Under the same standard occlusion value and rate, the higher the value of the tested pressure is, the longer the alarm time will be delayed.**
-

A.7 Infusion Accuracy Curve


The following typical infusion accuracy table expresses performance after infusion has started and infusion fluctuations occurring within a certain period of time after normal infusion flow volumes have been reached. The infusion accuracy table is for reference only; detailed infusion accuracy curve is in accordance with the final device.

Plotted on the basis of data collected over a two-hour measurement period.

Syringe brand: Dove
Syringe specification: 20ml
Sampling quantity of pump: 3
Sampling quantity of syringe: 3
Sampling rate: 1ml/h
Sampling interval: $\Delta t = 0.5$ min
Test period: $t = 120$ mins
Infusion rate: Q (m/h)

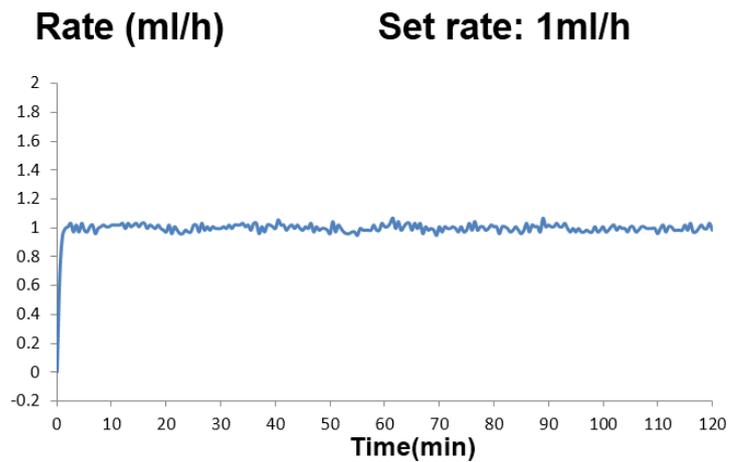

Syringe brand: Dove
Syringe specification: 20ml
Sampling quantity of pump: 3
Sampling quantity of syringe: 3
Sampling rate: 5ml/h
Sampling interval: $\Delta t = 0.5$ min
Test period: $t = 120$ mins
Infusion rate: Q (m/h)

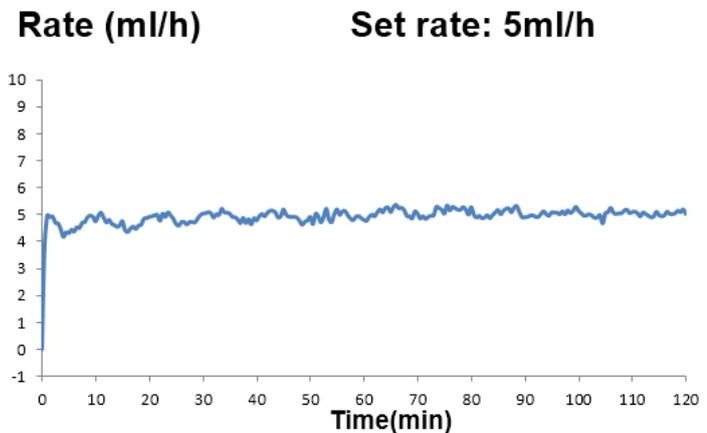

NOTE

- Infusion accuracy may be influenced by the pump's environment (such as pressure, temperature, humidity and any infusion consumables used).
-

A.8 Trumpet Curve

Flow rate deviation over time ($p\Delta t$)

Syringe brand: Dove

Syringe specification: 20ml

Sampling quantity of pump: 3

Sampling quantity of syringe: 3

Sampling rate: 1ml/h


Sampling interval: $\Delta t = 0.5$ min

Observation windows: $p\Delta t = 2, 5, 11, 19, 31$ mins


Maximum deviation over the course of a full observation window: EPmax (%)

Minimum deviation over the course of a full observation window: EPmin (%)

Average deviation: A (%)

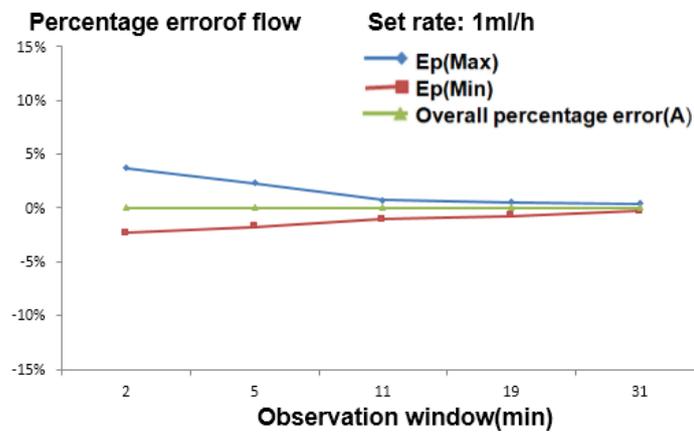

Syringe brand: Dove
Syringe specification: 20ml
Sampling quantity of pump: 3
Sampling quantity of syringe: 3
Sampling rate: 5ml/h
Sampling interval: $\Delta t = 0.5$ min
Observation windows: $p\Delta t = 2, 5, 11, 19, 31$ mins
Maximum deviation over the course of a full observation window: EPmax (%)
Minimum deviation over the course of a full observation window: EPmin (%)
Average deviation: A (%)

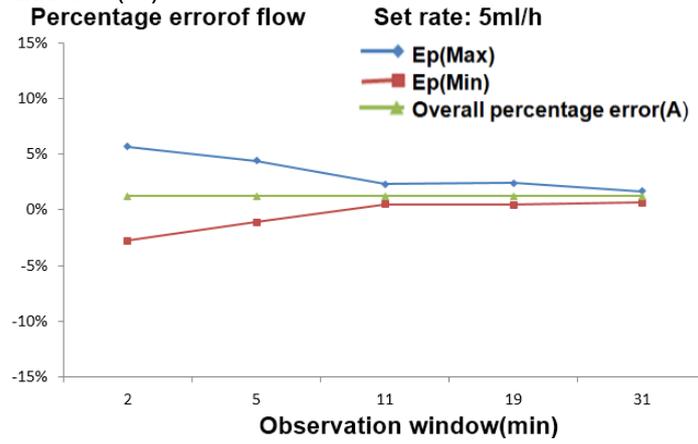

B EMC and Radio Regulatory Compliance

B.1 EMC

The device meets the requirements of IEC 60601-1-2:2014.

NOTE

- **Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this device could result in increased electromagnetic emissions or decreased electromagnetic immunity of this device and result in improper operation.**
 - **Use of this device adjacent to or stacked with other device should be avoided because it could result in improper operation. If such use is necessary, this device and the other device should be observed to verify that they are operating normally.**
 - **Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the this device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this device could result.**
 - **The non-ME EQUIPMENT (e.g. ITE) that is a part of an ME SYSTEM may be disrupted by the electromagnetic interference of nearby equipment. It may be necessary to take mitigation measures, such as re-orienting or relocating the non-ME EQUIPMENT or shielding the location.**
 - **This device is intended for use in professional healthcare facility environment only. If it is used in special environment, such as magnetic resonance imaging environment, the equipment/system may be disrupted by the operation of nearby equipment.**
-

Guidance and Declaration - Electromagnetic Emissions

The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment.

Emission test	Compliance	Electromagnetic environment - guidance
Conducted and radiated RF EMISSIONS CISPR 11	Group 1	The device uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic device.
Conducted and radiated RF EMISSIONS CISPR 11	Class A	The device is suitable for use in all establishments other than domestic and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic distortion IEC 61000-3-2	Class A	The device is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Voltage fluctuations and flicker IEC 61000-3-3	Complies	

NOTE

-
- **The device needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided below.**
 - **Other devices may affect this device even though they meet the requirements of CISPR.**
 - **The EMISSIONS characteristics of this device make it suitable for use in industrial areas and hospitals (CISPR 11 class A). If it is used in a residential environment (for which CISPR 11 class B is normally required) this device might not offer adequate protection to radio-frequency communication services. The user might need to take mitigation measures, such as relocating or re-orienting the device.**
 - **If the essential performance is lost or degraded, it may be necessary to take mitigation measures, such as re-orienting or relocating the ME EQUIPMENT or ME SYSTEM or shielding the location or stopping using the infusion pump system and contact the service personnel.**
-

If the device is operated within the electromagnetic environment listed in Table **Guidance and Declaration —Electromagnetic Immunity**, the system will remain safe and provide the following essential performance:

- **Operating mode**
- **Accuracy**
- **Function**
- **Protection against UNINTENDED BOLUS volumes**
- **Occlusion**
- **ALARM CONDITIONS regarded**
- **Data stored**

Guidance and Declaration - Electromagnetic Immunity			
The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±15kV air	±8 kV contact ±15kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV for input/output lines (length greater than 3 m)	±2 kV for power supply lines ±1 kV for input/output lines (length greater than 3 m)	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1 kV line(s) to line(s) ±2 kV line(s) to earth	±1 kV line(s) to line(s) ±2 kV line(s) to earth	
Voltage dips and Voltage interruptions IEC 61000-4-11	0 % U_T for 0,5 cycle 0 % U_T for 1 cycle and 70 % U_T for 25/30 cycles 0 % U_T for 250/300	0 % U_T for 0,5 cycle 0 % U_T for 1 cycle and 70 % U_T for 25/30 cycles 0 % U_T for 250/300	Mains power quality should be that of a typical commercial or hospital environment. If the user of our product requires continued operation during power mains interruptions, it is recommended that our

	cycle	cycle	product be powered from an uninterruptible power supply or a battery.
RATED power frequency magnetic fields IEC 61000-4-8	30 A/m 50 Hz / 60 Hz	30 A/m 50 Hz / 60 Hz	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
Note: U_T is the A.C. mains voltage prior to application of the test level.			

Guidance and Declaration - Electromagnetic Immunity			
The device is intended for use in the specified electromagnetic environment. The customer or the user of the device should assure that it is used in such an environment as described below.			
Immunity test	IEC 60601 Test level	Compliance level	Electromagnetic environment - guidance
Conducted disturbances induced by RF fields IEC61000-4-6	3 Vrms 150 kHz to 80 MHz	10 Vrms	Portable and mobile RF communications equipment should be used no closer to any part of the device, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance: $d = \left[\frac{3.5}{V} \right] \sqrt{P} \quad 150\text{k to } 80 \text{ MHz}$ $d = \left[\frac{3.5}{E} \right] \sqrt{P} \quad 80 \text{ MHz to } 800 \text{ MHz}$ $d = \left[\frac{7}{E} \right] \sqrt{P} \quad 800 \text{ MHz to } 2.7 \text{ GHz}$ where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m).
	6 Vrms in ISM bands and amateur radio bands ^a between 0,15 MHz and 80 MHz	6 Vrms	
Radiated RF EM fields IEC61000-4-3	10V/m 80 MHz to 2.7 GHz	10V/m	
Proximity fields from RF wireless communications equipment IEC61000-4-3	27 V/m 380–390 MHz	27 V/m	
	28 V/m 430–470 MHz, 800–960 MHz, 1700–1990 MHz,	28 V/m	

	2400–2570 MHz		Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey ^b , should be less than the compliance level in each frequency range ^c . Interference may occur in the vicinity of equipment marked with the following symbol:  .
	9 V/m 704–787 MHz, 5100–5800 MHz	9 V/m	
Note 1: At 80 MHz and 800 MHz, the higher frequency range applies. Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			
<p>^a The ISM (industrial, scientific, and medical) bands between 150 kHz and 80 MHz are 6.765 MHz to 6.795 MHz; 13.553 MHz to 13.567 MHz; 26.957 MHz to 27.283 MHz; and 40.66 MHz to 40.70 MHz. The amateur radio bands between 0,15 MHz and 80 MHz are 1,8 MHz to 2,0 MHz, 3,5 MHz to 4,0 MHz, 5,3 MHz to 5,4 MHz, 7 MHz to 7,3 MHz, 10,1 MHz to 10,15 MHz, 14 MHz to 14,2 MHz, 18,07 MHz to 18,17 MHz, 21,0 MHz to 21,4 MHz, 24,89 MHz to 24,99 MHz, 28,0 MHz to 29,7 MHz and 50,0 MHz to 54,0 MHz.</p> <p>^b Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the device is used exceeds the applicable RF compliance level above, the device should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the device.</p> <p>^c Over the frequency ranges 150 kHz to 80 MHz, field strengths should be less than 3V/m.</p>			

Recommended Separation Distances between Portable and Mobile RF, Communications Equipment and This Equipment			
The equipment is intended for use in an electromagnetic environment in which radiated RF disturbance are controlled. The customer or the user of the device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the device as recommended below, according to the maximum output power of the communication equipment.			
Rated Maximum Output power of Transmitter Watts (W)	Separation Distance According to Frequency of Transmitter (m)		
	150 kHz to 80 MHz $d = \left[\frac{3.5}{V} \right] \sqrt{P}$	80 MHz to 800 MHz $d = \left[\frac{3.5}{E} \right] \sqrt{P}$	800 MHz to 2.7 GHz $d = \left[\frac{7}{E} \right] \sqrt{P}$
0.01	0.04	0.04	0.07
0.1	0.11	0.11	0.22
1	0.4	0.4	0.7
10	1.1	1.1	2.2
100	4	4	7
For transmitters at a maximum output power not listed above, the recommended separation distanced in meters (m) can be determined using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer. Note 1: At 80 MHz and 800 MHz, the higher frequency range applies. Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			

B.2 Radio Regulatory Compliance

RF Parameter

Radio devices	IEEE 802.11b/g/n (2.4GHz Wi-Fi)
Operating frequency	2412MHz to 2484MHz
Modulation mode	CCK, OFDM
Output power	≤20dBm



The radio device used in this product is in compliance with the essential requirements and other relevant provisions of Directive 2014/53/EU.

C Default Factory Settings

This chapter presents some default factory settings. User cannot change the factory default, but may restore to the default factory settings when it is necessary.

C.1 Alarms

Alarm Setting	Factory Default
Volume	4
Alarm sound	Sound2

C.2 Interface

UI	Factory Default
Brightness	4

C.3 Parameters

Setting Parameters	Factory Default
KVO Rate	0.5ml/h
Pressure unit	mmHg
Occl. pressure	Medium (525mmHg)
Auto-lock	Off
Reminder	2min
Near end	3min
Commonly used brand	Latest brand library and 2 other brands
Bed No.	---
Standby time range	24:00

C.4 System Time

System time and date	Factory Default
Time	00:00
Date	01/01/2015

D Alarm Information

This chapter presents the alarm information of the veterinary syringe pump. Prompt information for operation guidance will not be presented in this chapter.

The table shows the appropriate countermeasures for each piece of information related to alarm triggering. If the problem still exists after operating according to the countermeasures, please contact the company.

Alarm Information	Alarm Level	Reason	Countermeasure
[Occlusion]	High	Infusion cannula blocked during infusion, and occlusion pressure reached the threshold of preset occlusion pressure threshold.	Press  to cancel alarm, clean the cannula occlusion, and then press  to continue the infusion.
[VTBI Done]	High	Infusion volume reached the preset VTBI.	Press  to cancel alarm.
[KVO Finish]	High	Alarm is triggered when KVO model runs 30 minutes unattended.	Press  to cancel alarm.
[Battery Empty]	High	Only powered with built-in battery, battery is empty.	Connect to the AC power source, press  or  to cancel alarm.
[System Error]	High	Motor operation error, data communication error, sensor failure etc.	Alarm cannot be cancelled. Please stop operation and contact the company.
[Syringe Disengaged]	High	Syringe is disengaged during the operation of veterinary syringe pump.	Press  to cancel alarm.
[Syringe Empty]	High	The syringe is empty during the infusion.	Press  to cancel alarm.
[System abnormal]	Mid-level	Charging circuit error, supplying circuit error and etc.	Alarm cannot be cancelled. Please stop operation and contact the company.

Alarm Information	Alarm Level	Reason	Countermeasure
[Standby Time Expired]	Mid-level	Standby complete.	Press  to cancel alarm, and then exit standby or continue standby by pressing  .
[Battery Low]	Low	Only powered with built-in battery, battery charge is insufficient.	Connect to the power source to cancel alarm automatically.
[AC Power Disconnection]	Low	Power cord disconnected when the network power source supply is powered.	Connect to the AC power or external DC power source to cancel the alarm automatically.
[Reminder]	Low	The veterinary syringe pump performs no operation during the set reminder time (except standby status and calibration status) after the syringe is installed to it.	Operate the pump to cancel alarm.
[VTBI Near Done]	Low	Required time for the remaining VTBI volumes almost reaches the [Near End] time set by the users.	1. The alarm will not be cancelled automatically until the infusion is completed, and then switch to [VTBI Done] alarm. 2. Or press  to pause alarm sound.
[Syringe not inserted]	Low	Start infusion without inserting syringe or inserting syringe incorrectly.	Insert syringe correctly.
[No communication]	Low	Veterinary syringe pump and BeneFusion CS5 Infusion Supervision System are communicated successfully over Wi-Fi, the network communication is abnormally interrupted for 3 minutes. After the alarm is triggered, infusion of the pump will not be influenced, and the pump continues infusion.	Press  or restore the communication between veterinary syringe pump(s) and BeneFusion CS5 Infusion Supervision System.

Alarm Information	Alarm Level	Reason	Countermeasure
[Syringe Nearly Empty]	Low	Required time for the remaining liquid within the syringe almost reaches empty time.	1. The alarm will not be cancelled automatically until the syringe is empty. 2. Or press  to pause alarm sound.

NOTE

- All alarm sounds can be paused by pressing , except for the circumstance of [Battery Empty].

FOR YOUR NOTES

E Symbols and Terms

E.1 List of Units

Abbreviation	Meaning
A	ampere
°C	centigrade
cm	centimeter
dB	decibel
g	gram
hr	hour
Hz	hertz
inch	inch
k	kilo
kg	kilogram
kPa	kilopascal
l	litre
lb	pound
m	meter
mg	milligrams
min	minute
ml	milliliter
mm	millimeters
mmHg	millimeters of mercury
s	second
µg	Microgram
V	volt
VA	volt ampere
W	watt

E.2 List of Symbols

Symbols	Meaning
-	minus
%	percent
/	Per; divide; or
~	to
^	power
+	plus
=	equal to
<	less than
>	greater than
≤	less than or equal to
≥	greater than or equal to
±	plus or minus
×	multiply
©	copyright

E.3 List of Terms

Abbreviation	Meaning
AC	Alternating current
Anti-Bolus	Anti-Bolus
BOLUS	Bolus
CCU(CICU)	Cardiac Intensive Care Unit
CE	Conformité Européenne
CISPR	International Special Committee on Radio Interference
CPU	central processing unit
DC	Direct current
DPS	Dynamic Pressure System
ECU(EICU)	Emergency Intensive Care Unit

Abbreviation	Meaning
EEC	European Economic Community
EMC	Electromagnetic compatibility
EMI	Electromagnetic interference
EtO	C2H4O
ICU	Intensive Care Unit
ID	Identification
IEC	International Electrotechnical Commission
IEEE	Institute of Electrical and Electronic Engineers
ISO	International organization for Standardization
KVO	Keep vein open
LED	light emitting diode
Max	Maximum
MDD	Medical Device Directive
Min	Minimum
MRI	magnetic resonance imaging
N/A	not applied
OR	operating room
SN	Series Number
TIVA	Total Intra Venous Anesthesia
VTBI	Volume To Be Infused

E.4 List of Unit Conversion

Unit Symbols	Unit Conversion
kPa	1kPa=7.5mmHg=0.145psi=0.01bar
psi	1psi=51.724mmHg=6.897kPa=0.069bar
bar	1bar=750mmHg=14.5psi=100kPa

FOR YOUR NOTES

F Toxic and Hazardous Substances or Elements

Name of the Parts		Pb	Hg	Cd	Cr(VI)	PBB	PBDE
		Pb	Hg	Cd	Cr(VI)	PBB	PBDE
Device housing	Front housing	○	○	○	○	○	○
	Back housing	○	○	○	○	○	○
	Keys	○	○	○	○	○	○
	Facing	○	○	○	○	○	○
	Labels	○	○	○	○	○	○
Display	Display	○	○	○	○	○	○
Host	Host hardware	○	○	○	○	○	○
	Internal cables	○	○	○	○	○	○
	PCBA	○	○	○	○	○	○
Packaging	Cartons (K=K crimp paper)	○	○	○	○	○	○
	Foam packages (EPE)	○	○	○	○	○	○
	Plastic bag (PE)	○	○	○	○	○	○
General	Connecting pieces	○	○	○	○	○	○
	Power cord	○	○	○	○	○	○
Battery	Battery	○	○	○	○	○	○
Accessories	Accessories	○	○	○	○	○	○
Remark	<p>○: Indicates that this toxic or hazardous substance contained in all of the homogeneous materials for this part is below the limit requirement in Directive 2011/65/EU.</p> <p>✕: Indicates that this toxic or hazardous substance contained in at least one of the homogeneous materials used for this part is above the limit requirement in Directive 2011/65/EU.</p>						

P/N: 046-00036A-00(2.0)

GERMANY

EICKEMEYER KG
Eltastraße 8
78532 Tuttlingen
T +49 7461 96 580 0
F +49 7461 96 580 90
info@eickemeyer.de
www.eickemeyer.de

SWITZERLAND

EICKEMEYER AG
Sandgrube 29
9050 Appenzell
T +41 71 788 23 13
F +41 71 788 23 14
info@eickemeyer.ch
www.eickemeyer.ch

POLAND

EICKEMEYER Sp. z o.o.
Al. Jana Pawła II 27
00-867 Warszawa
T +48 22 185 55 76
F +48 22 185 59 40
info@eickemeyer.pl
www.eickemeyer.pl

ITALY

EICKEMEYER S.R.L.
Via G. Verdi 8
65015 Montesilvano (PE)
T +39 085 935 4078
F +39 085 935 9471
info@eickemeyer.it
www.eickemeyer.it

DENMARK

EICKEMEYER ApS
Solbakken 26, Hammelev
6500 Vojens
T +45 7020 5019
info@eickemeyer.dk
www.eickemeyer.dk

NETHERLANDS

EICKEMEYER B.V.
Bellweg 44
4104 BJ Culemborg
T +31 345 58 9400
info@eickemeyer.nl
www.eickemeyer.nl

UNITED KINGDOM

EICKEMEYER Ltd.
3 Windmill Business Village
Brooklands Close
Sunbury-on-Thames
Surrey, TW16 7DY
T +44 20 8891 2007
info@eickemeyer.co.uk
www.eickemeyer.co.uk

CANADA

EICKEMEYER Inc.
617 Douro Street, Suite #205
Stratford, Ont. Canada
N5A 0B5
T +1 519 273 5558
F +1 519 271 7114
info@eickemeyervet.ca
www.eickemeyercanada.ca